



FINDING YOUR
PERFECT FIT

Guide to measure and choose
the right size of your Neoflex
postural support solution.



Neo 00 Universal

- Neo 00.01 Multi-use band
- Neo 00.02 Multi-use band extension

Neo 20

- Neo 20.01 Chest support - wide
- Neo 20.02 Chest support with zipper - wide
- Neo 20.03 Chest support - slim
- Neo 20.04 Chest support with zipper - slim
- Neo 20.05 Shoulder harness
- Neo 20.06 Linked shoulder harness
- Neo 20.07 Chest belt

Neo 30

- Neo 30.01 2 points hip belt
- Neo 30.02 4 points hip belt
- Neo 30.03 Abduction control

Neo 40

- Neo 40.01 Calf band
- Neo 40.02 Ankle control



HOW TO MEASURE?

Neo 00.01 Multi-use band

Neo 00.02 Multi-use band extension

Measure the total chest + backrest circumference, add an extra 15-20 cm, and report this dimension to the table below to find the right size.



A	Size
<80 cm	S
80-120 cm	M
121-140 cm	L
141-160 cm	XL

HOW TO MEASURE?

Neo 20.01 Chest support - wide

Neo 20.02 Chest support with zipper - wide

Neo 20.03 Chest support - slim

Neo 20.04 Chest support with zipper - slim

Measure the chest width under the arms, and report this dimension to the table below to find the right size.



B	Size
>24 cm	XS
25-26 cm	S
27-32 cm	M
33-39 cm	L
40-46 cm	XL

HOW TO MEASURE?

Neo 20.05 Shoulder harness

Neo 20.06 Linked shoulder harness



Measure the total length from **top of the shoulder to the elbow base** and report this dimension to the table below to find the right size.



A	Size
25-30 cm	S
31-37 cm	M
38-40 cm	L
41-45 cm	XL

HOW TO MEASURE?

Neo 30.01 2 points hip belt
Neo 30.02 4 points hip belt

Measure the pelvis width and report this dimension to the table below to find the right size.





A	Size
25-33 cm	XS
31-40 cm	S
39-52 cm	M
48-65 cm	L

HOW TO MEASURE?

Neo 30.03 Abduction control

Measure the distance from the seating surface to the outer part of the thigh as shown on the picture, and report this dimension to the table below to find the right size.

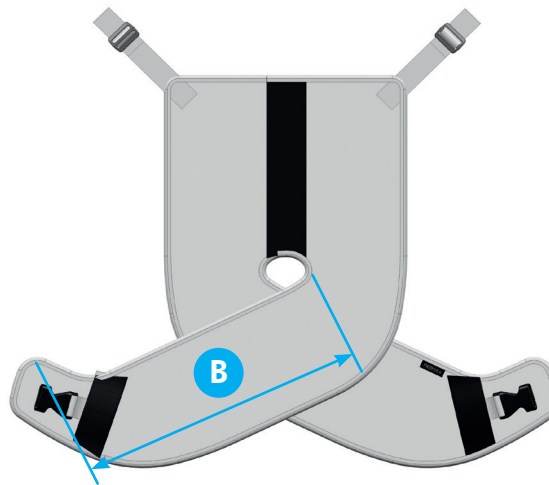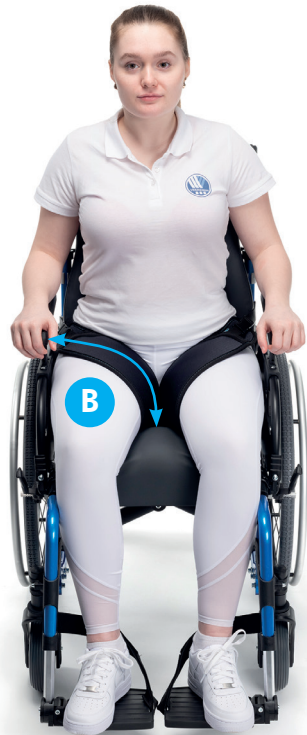

B	Size
17-24 cm	XS
24-26 cm	S
27-32 cm	M
33-38 cm	L

HOW TO MEASURE?

Neo 40.01 Calf band

Measure the distance between the footrest main tubes, and report this dimension to the table below to find the right size.



A	Size
18-24 cm	S
25-32 cm	M
33-40 cm	L
40-50 cm	XL

HOW TO MEASURE?

Neo 40.02 Ankle control for both feet

Measure ankle perimeter and report this dimension to the table below to find the right size.



	Size
14-17	XS
17-20	S
19-23	M
22-29	L
28-33	XL

