


REV
ONE SPRING

18.08.2025

01.07.2026

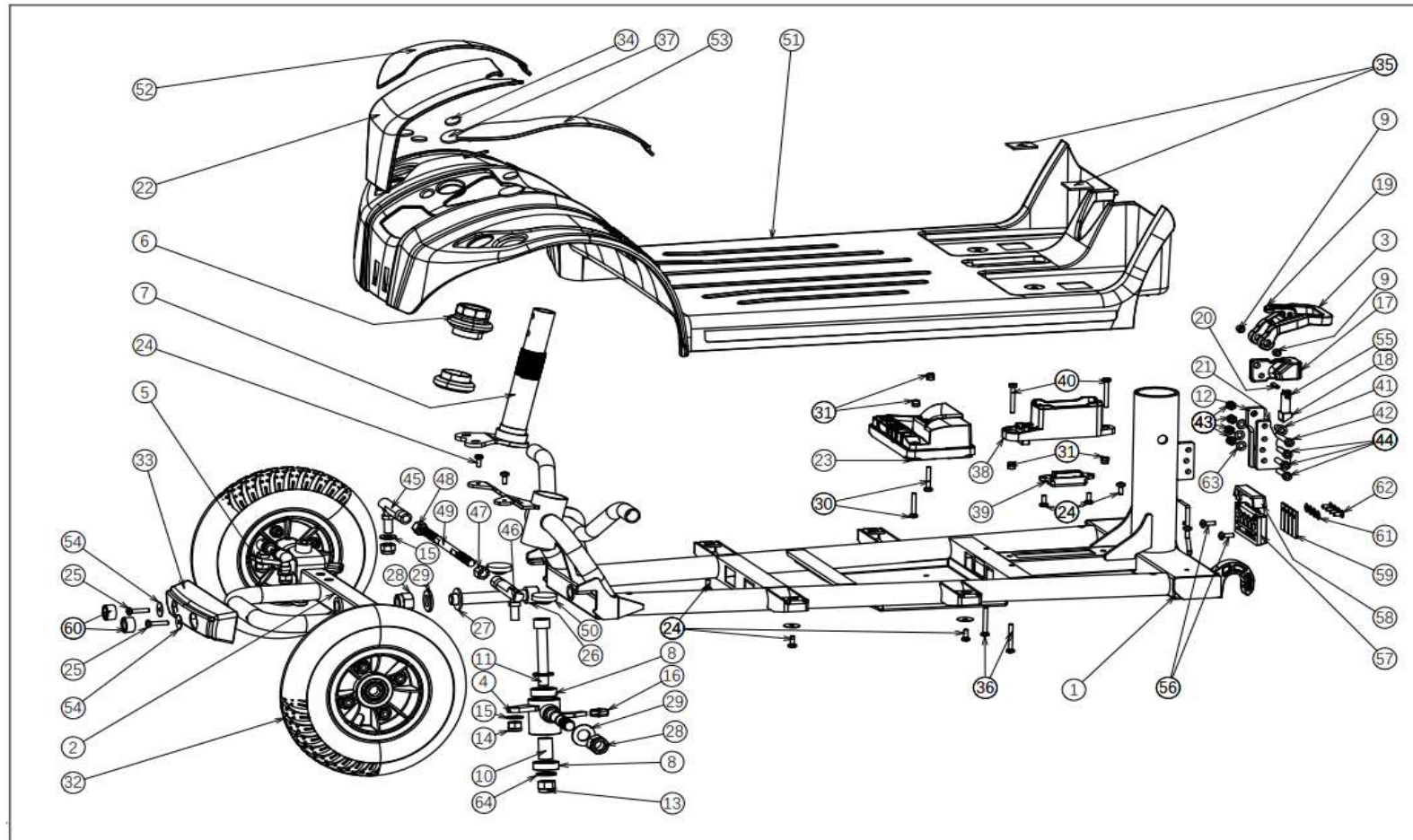


no	part nu	part	QT	no	part nu	part	QT
1	1920656	MAIN FRAME	1	36			
2	1918862	FRONT FORK	1	37			
3				38	1920657	BATTERY CONNECTOR HOLDER 1	1
4	1920340	FRONT AXLE L		39	1920658	BATTERY CONNECTOR HOLDER 2	1
5	1920341	FRONT AXLE R			1922814	BATTERY CONNECTOR +CABLE	1

6				40			
7				41			
7				42			
7				43			
7				44	1918857	BALL JOINT LE	
8				46	1918858	BALL JOINT RIGHT	
8				45			
8				47			
8				48	1918859	CONNECTION ROD	
9			1	49			
10				50	1918851	FOOT PLATE	1
11				51	1920443	SET COLOR PANEL FRONT green	
				52	1920444	SET COLOR PANEL FRONT orange	
				52	1920445	SET COLOR PANEL FRONT Blue	
				52	1920446	SET COLOR PANEL FRONT pink	
			1	52			
13				53	1920634	WASHER M4 NF E25-513	
14				54			
15			2	55			
16				56	1918861	FRONT CONTACT POINT HOLDER	1
17				57			
18			2	58	1918860	CONTACT POINT FRONT	4
19				59	1920635	BOLT CAP	6
20				60			
21				61			
22	1918852	FRONT COVER	1	62			
23	1919090	POWER MODULE	1				
24	1918856	STEERING AXLE ONE	1				
25	1920633	BOLT M4X20 DIN 7985	13				
26							
27							
28							
29							
30							


no	part nu	part	QT		part nu	part	QT
not depicted	1920394	TIRE	4	no			
not depicted	1920393	INNER TIRE	4				
1							
not depicted	1922014	BUSH TRANSMISION .	1	28			
	1920396	MOTOR	1	29			
	1920397	MOTORBRAKE	1	30	1922668	SPRING	2
2	1918887	DIFF.+MOTOR+REM	1	31			
	1920395	DIFF	1	32			
3	1922523	WHEEL ANTI TIPPING	2	33			
4				34			
5				35			
6				36			
7				37	1922190	RED MOTORBRAKE LEVER COVER	1
8	1922588	REAR COVER SPRING	1	38	1920439	COLOR PANEL SET L/R green	
9	1922669	REAR UPPER COVER SPRING	1	0	1920440	COLOR PANEL SET L/R orange	
10				0	1920441	COLOR PANEL SET L/R Pink	
11				0	1920442	COLOR PANEL SET L/R blue	
12				0			
13				40			
14	1922549	BOLT ANTI TIP	2	41			
15	1922403	SPRING WASHER	4	42			
16				43			
17	1918889	REAR CONTACT POINT HOLDER	1	44			
18				45			
19							
20	1918890	CONTACT POINT REAR	4				
21							
22							
23	1922667	UPPER REAR FRAME	1				
24	1922665	LOWER REAR FRAME	1				
25	1922938	REAR WHEEL LU	2				

26	1922551	LOCK NUT	2
27			
not depicted	1923143	AXLE KEY	2

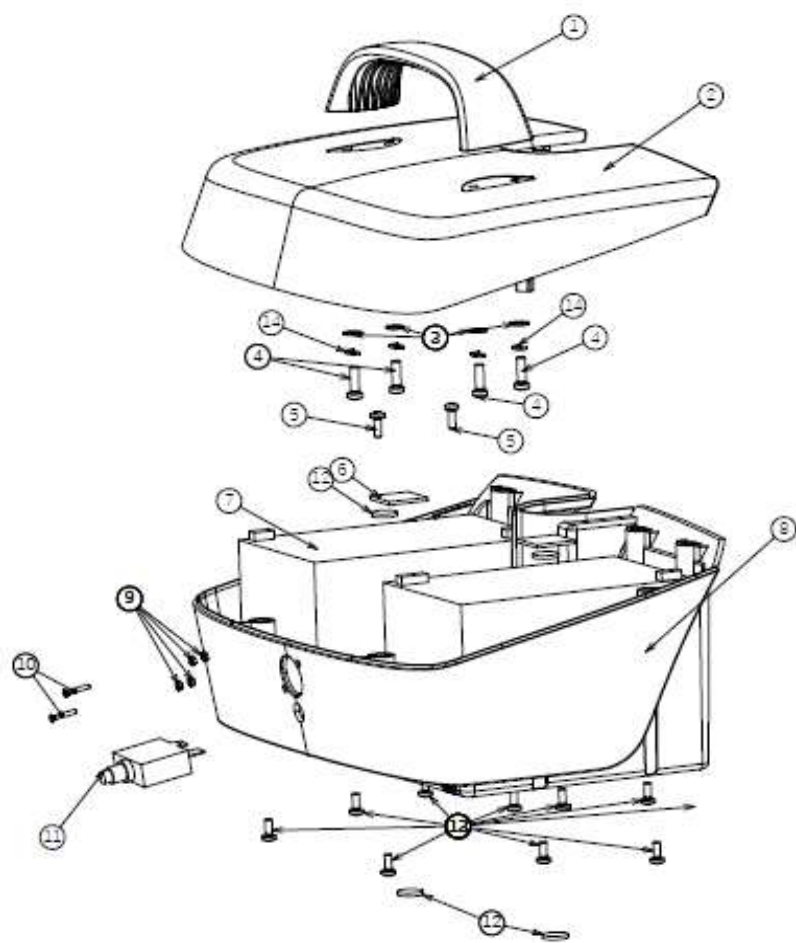


no	part nu	part	QT				
0	1922828	BATTERY BOX WITHOUT BATTERIES	1				
	1922929	BATTERY BOX COMPL	1				
0	1922672	CHARGER PLUG BATT BOX	1				
1							

2							
3							
4							
5							
6							
7		SI*BATT					
8							
9							
10							
11	1920531	MAIN FUSE	1				
12	1922784	SCREWMx10	5				
13							
14							
15			2				
16							
17							
18							
	1922776	CHARGER LITHIUM	1				

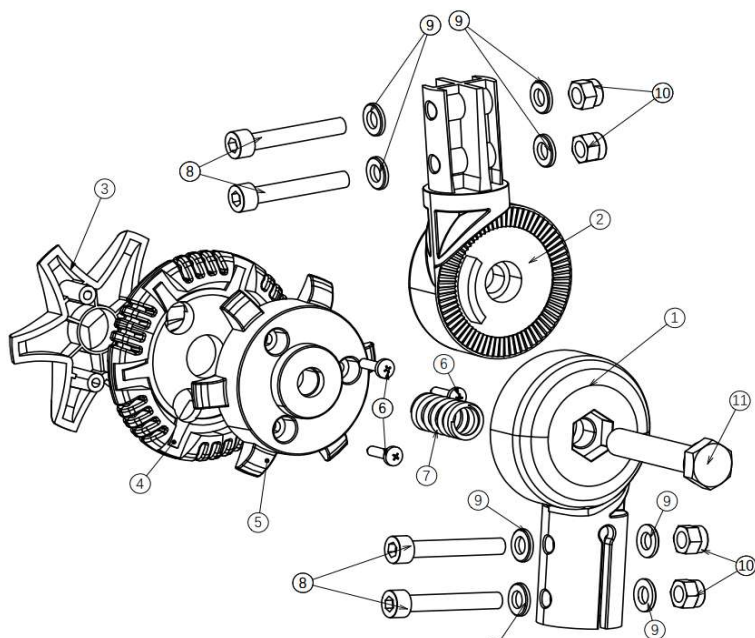


no	part nu	part	QT				
1	1921317	lower parrrt angle adujustment					
2							
3	1921320	outer knob angle adjust	1				
4	1921319	middle knob angle adjust	1				
5	1921318	inner knob angle adjust	1				
6	1921322	bolt M5 X 16					
7	1921321	spring knob angle adjust	1				
8							
9							
10							
11	1922522	bolt angle adjust M10 X55	1				
not depicted	1922514	Set 1/12	1				

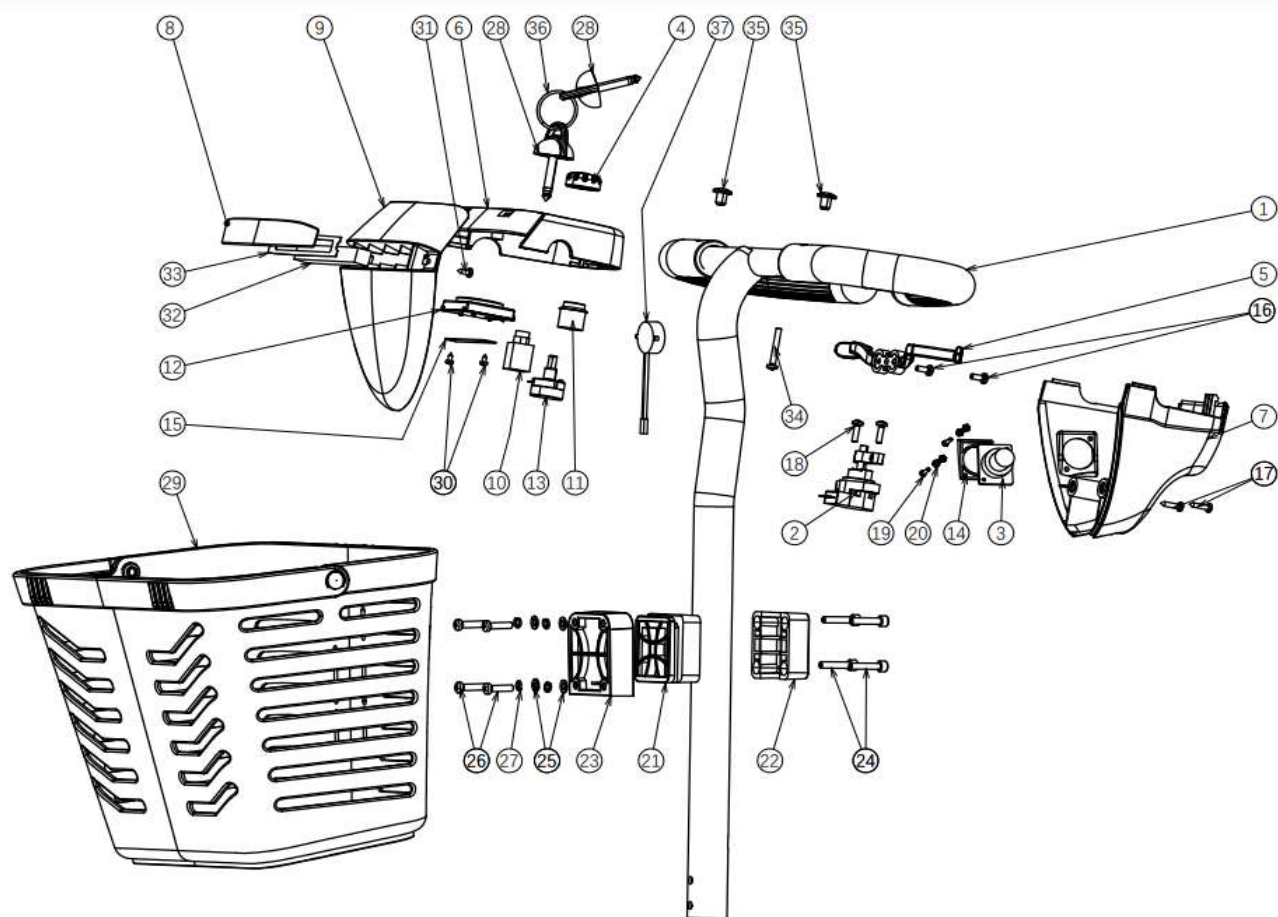

no	part nu	part	QT
1			
2	1918911	SPEED DIAL ASSEMBLY	1
3	1922674	CHARGER SOCKET	1
4	1924415	SPEED KNOB	1
5	1918910	SPEED LEVER	1
6	1923868	TOP COVER	1
7	1918913	STEERING COVER BACK	1
8	1921463	FRONT LIGHT COVER	1

9	1918914	STEERING COVER FRONT	1	
10	1919437	CONTACT IGNITION	1	
11	1920423	HORN SWITCH	1	
12	1918912	BATTERY INDICATOR	1	
13	1918909	POT SPEED	1	
14				
15				
16				
17				
18				
19				
20				
21	1920502	ASSEMBLY PLATE BASKET MIDDLE	1	
22	1920510	ASSEMBLY PLATE BASKET REAR	1	
23	1920501	ASSEMBLY PLATE BASKET FRONT	1	
	1920832	BASKET HOLDER COMPLETE		
24	1920511	BOLT M5X25	2	
25	1920503	WASHER	4	
26	1920505	BOLT M5X20	4	
27				
28	1918908	IGNITION KEY	1	
29	1920509	BASKET	1	
30				
31				
32	1920629	FRONT LIGHT	1	
33				
34				
35				
	1918977	MIRROR LEFT	1	
	1918978	MIRROR RICHT	1	
37	1920628	HORN	1	

12				
13	1922401	MAIN CABLE	1	
14				
15				
16				
17	1922672		1	
	1922773	4P PROGRAM ADAPTOR	1	

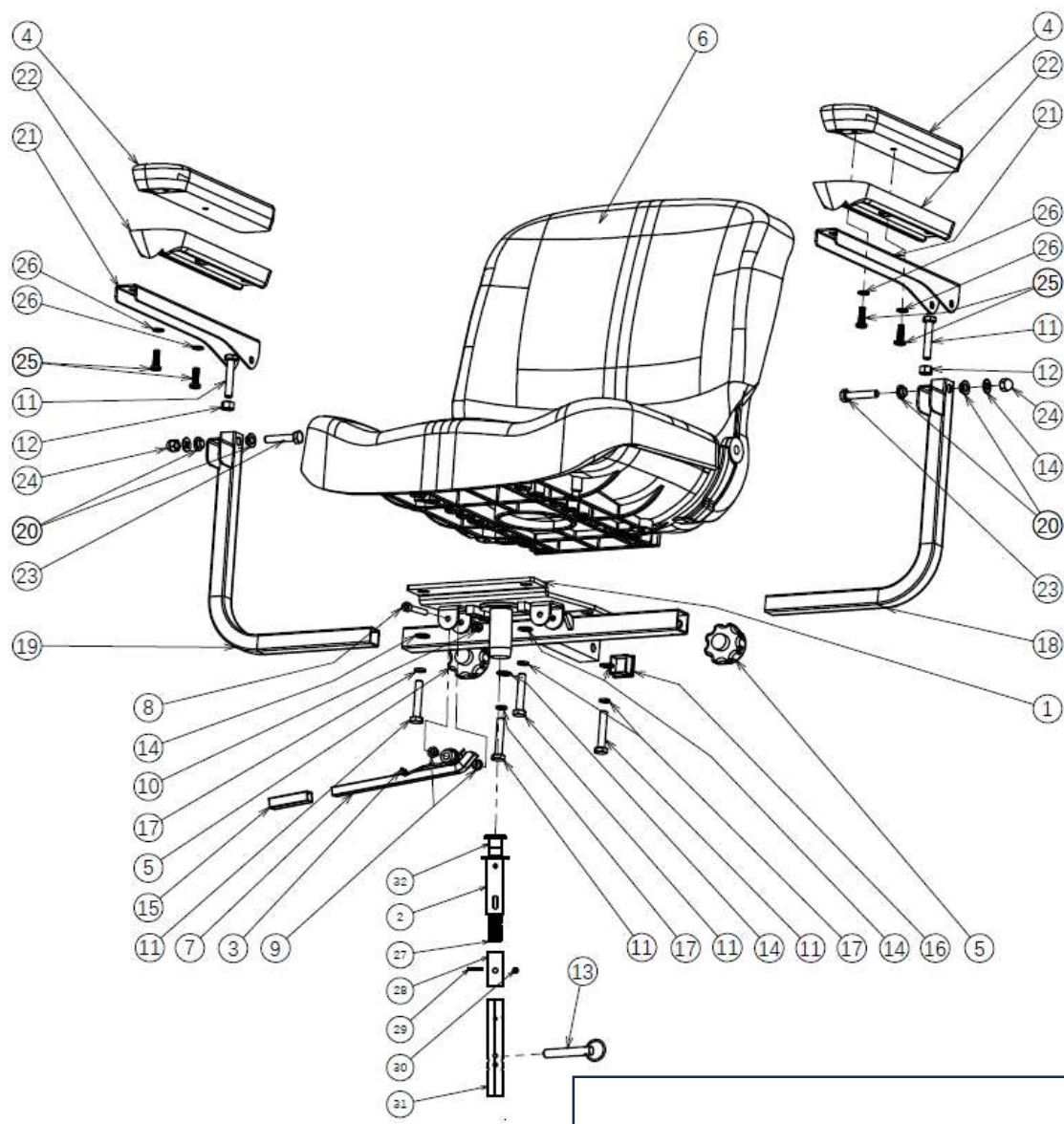

no	part nu	part	QT			
0						
1						

2						
3						
4	1918895	ARM PAD	2			
5	1920266	BOLT ARMREST FIXATION	2			
6	1918899	CHAIR	1			
6.1	1920640	BACK	1			
6.2	1920641	SEAT				
7	1920645	SEAT HANDLE	1			
NOT DEPICTED	1923200	SPRING SEAT HANDLE	1			
8						
8						
9						
10						
11	1924043	BOUT M8 X 45 DIN931	4			
12						
13						
14	1924044	RONDSEL M8 DIN126	4			
15						
16						
17	1924045	RONDSEL VEER M8DIN127B				
18	1918903	ARMREST LOWER LEFT	1			
19	1918907	ARMREST LOWER RIGHT	1			
20						
21	1918931	ARMRESR UPPER PART	2			
22						
23						
24						
	1920830	ARM REST COMPLETE L	1			
	1920831	ARM REST COMPLETE R	1			
	1921438	SEAT PLATE COMPL W/A tube	1			
	1922964	SEAT TUBE SPRING COMPL	1			