


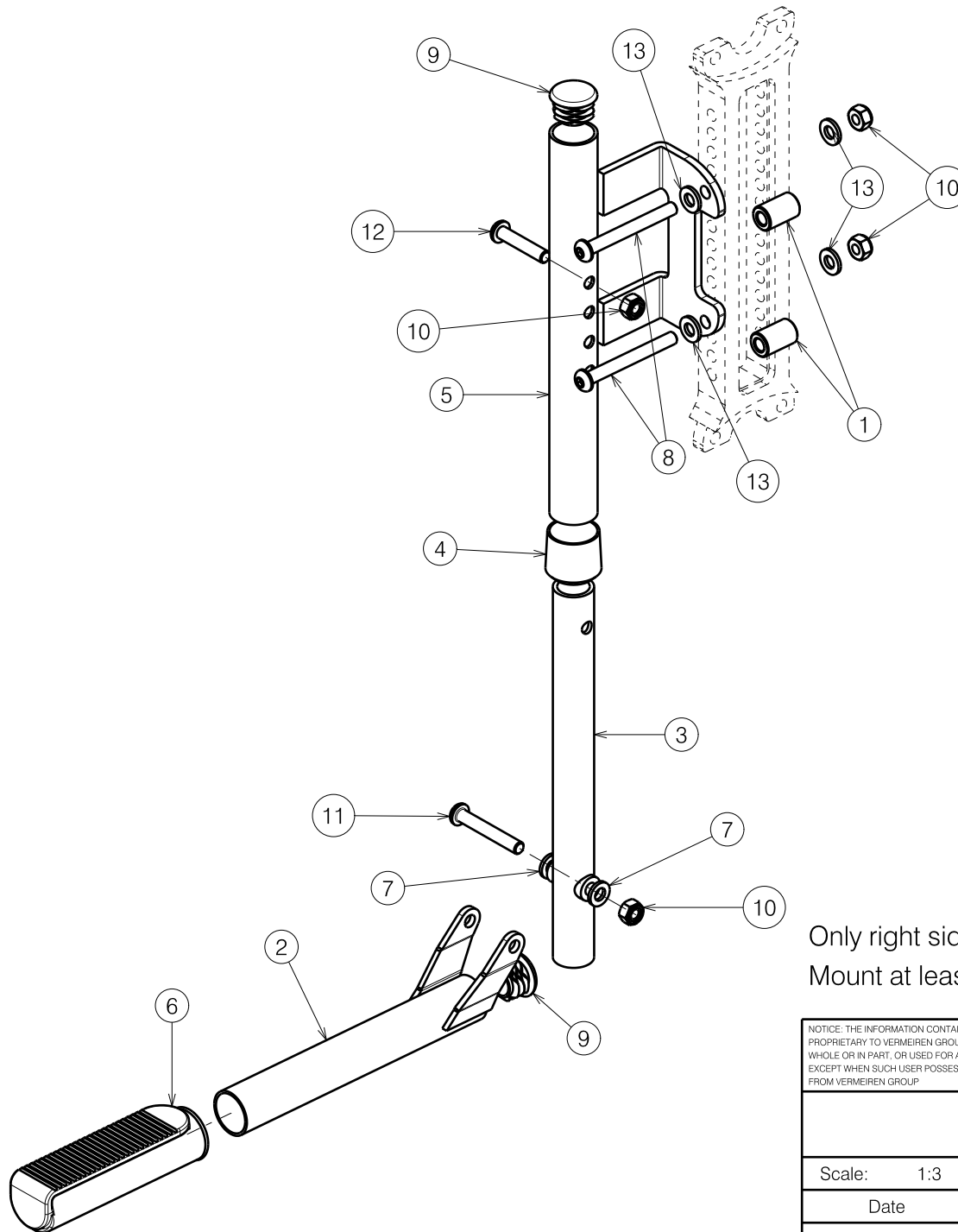

No.	Plan	Article	Qty
1	P00074685	GP30054	2
2	P00126699	2926154	1
3	P00126700	2926254	1
4	Y004359	F000181	1
5	YP08202 (R) YP08216 (L)	2610054 2610154	1
6	X003596	1998528	1
7	M6 DIN125 (Plast Black)	2994068	2
8	M6x55 ISO7380 (Black)	2994376	2
9	Cap D22	1990648	2
10	M6 DIN985 (Black)	2994063	4
11	M6x40 ISO7380 (Black)	2994066	1
12	M6x30 ISO7380 (Black)	2994129	1
13	Washer M6 DIN125A (Black)	2994414	4

Only right side depicted
Mount at least 8 cm above ground

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

TIPPING AID / TRIGO				---
				Plan No.
				E00129500/A.1
Scale: 1:3	Drafted	Verified	Confirmed	
Date	09.06.2026	12/06/2026	12/06/2026	
Initials	DZ	AZ	AS	