


use with WEICONLOCK AN 302-50

use with grease

Tighten with 36 Nm

Tighten with 36 Nm

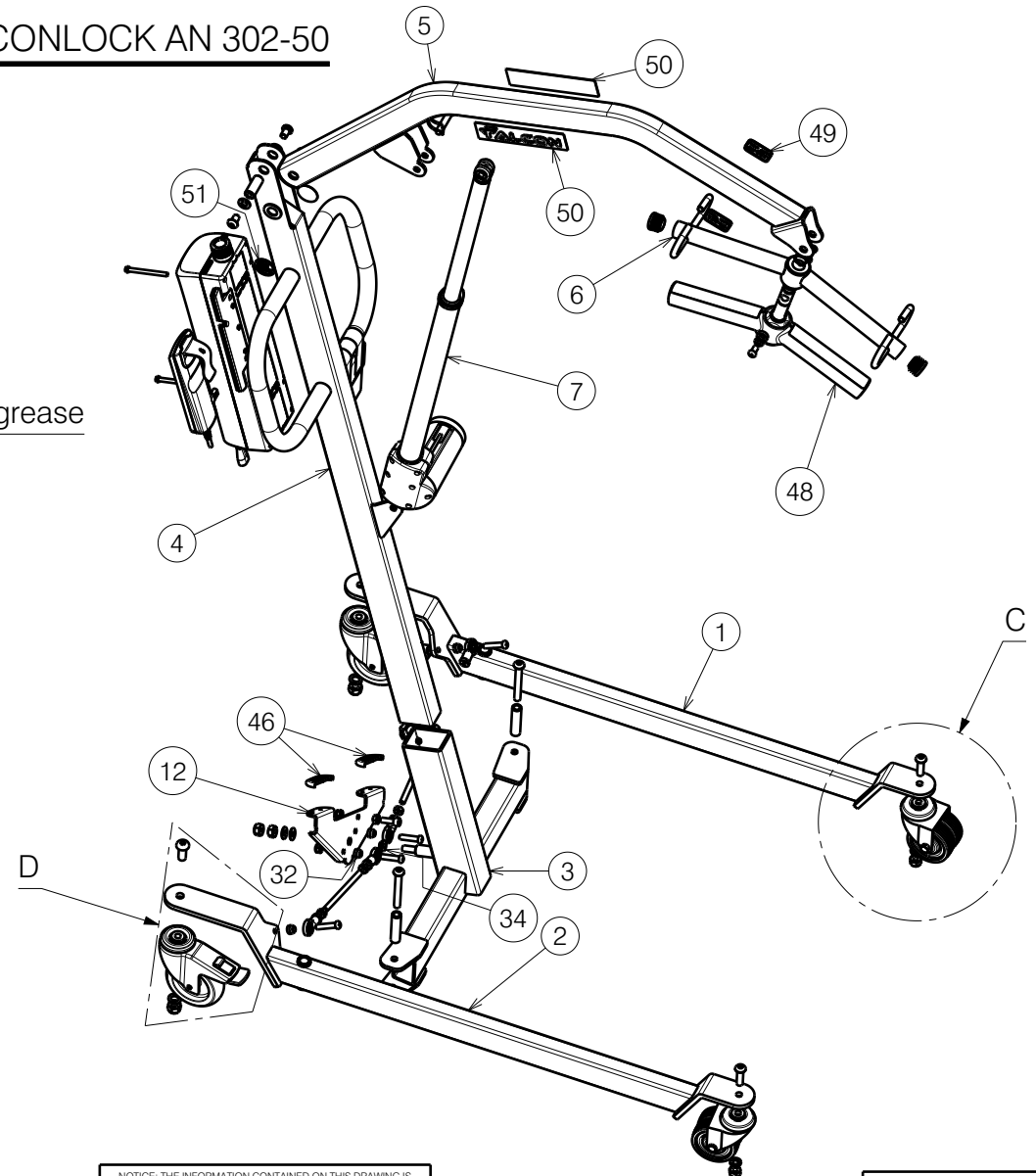

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/4

8003207

Plan No.


E00042257/J1.1

PATIENT HOIST FALCON


Scale: 1:12	Drafted	Verified	Confirmed
Date	10/08/2022	22/06/2026	22/06/2026
Initials	MK	PM	MK




A (1:3)

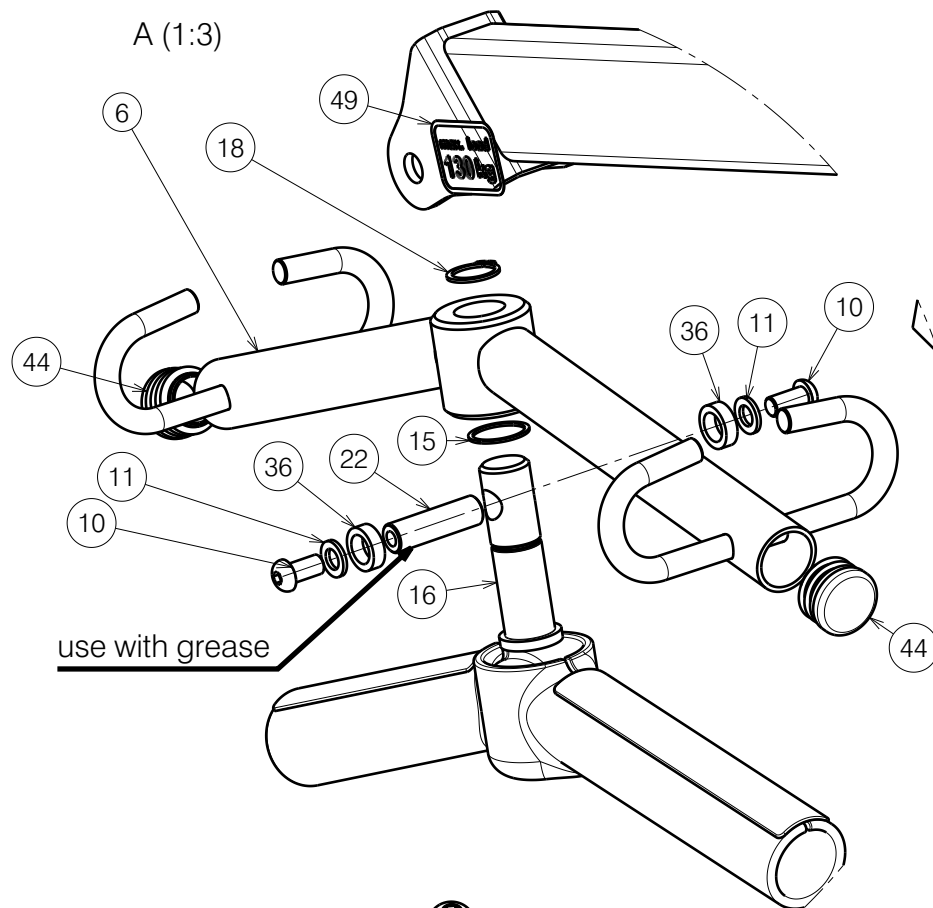


B (1:3)

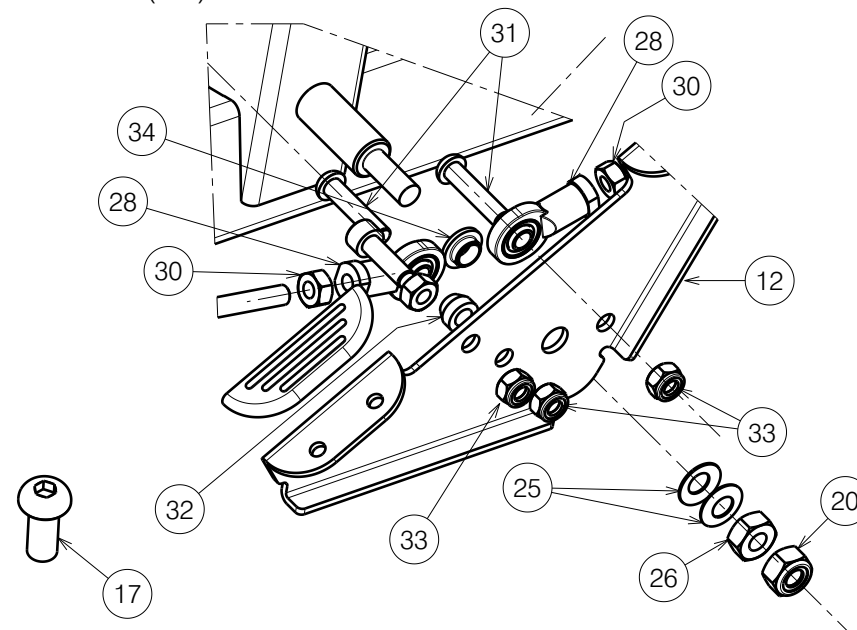

D (1:3)

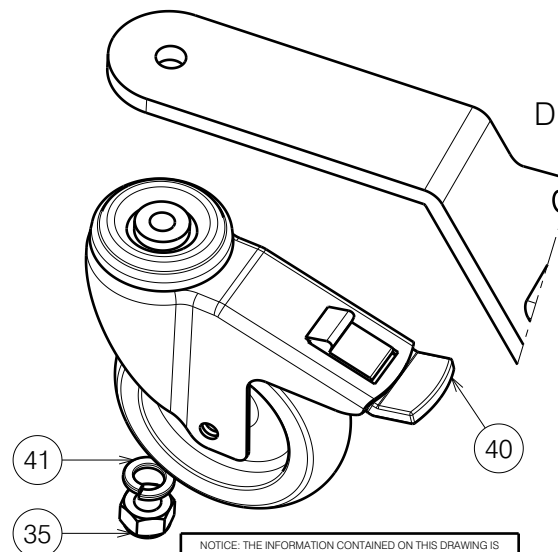

C (1:3)

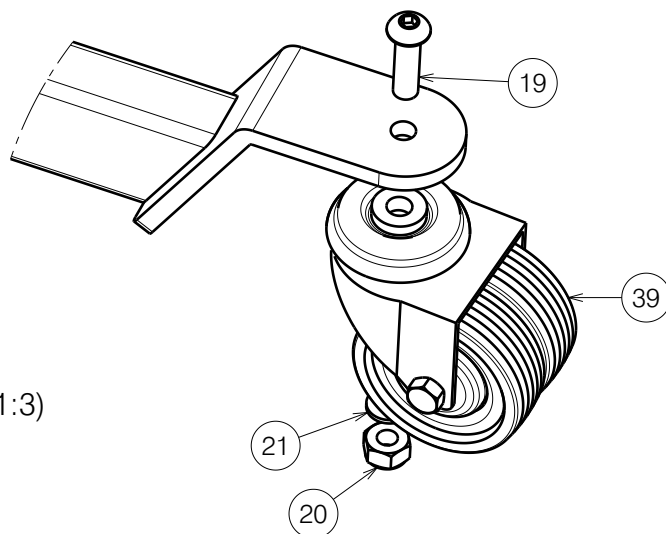

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/4 8003207

PATIENT HOIST FALCON

Plan No.

E00042257/J1.1

Scale:	1:12	Drafted	Verified	Confirmed
Date	10/08/2022	22/06/2026	22/06/2026	
Initials	MK	PM	MK	



No.	Plan	Article	Qty	25	Saucer spring	12296	2
1	P00036542	2657819	1	26	Nut M10 DIN934	1001672	1
2	P00038162	2657719	1	27	X004408	1711500	2
3	P00041195	2658119	1	28	Knee joint M8	1006835	4
4	P00041503	2658219	1	29	M8x40	1008739	2
5	P00041510	2658319	1	30	Nut M8 DIN934	1001671	4
6	P00038178	2658419	1	31	1993236	1993236	2
7	P00041119	1918901	1	32	P00074772	2098400	4
8	P00040355	GP28200	1	33	X001016	1001682	5
9	M10x16	1903370	2	34	XP01931	2993584	1
10	1991858	1991858	2	35	Nut M12 DIN985	1001684	2
11	Washer M8 DIN125B	1001556	2	36	XP03232	GP26300	2
12	P00078077	2098654	1	37	P00042258	1918904	1
13	Sterknop M8	1001580	1	38	P00042263	1918902	1
14	1001558	1001558	2	39	Wheel D75x50	1902764	2
15	Y016681	F006290	1	40	Wheel D100x32	1001252	2
16	P00041505	GP28100	1	41	M12 spring washer	1001572	2
17	M12x30	2995568	2	42	P00078081	G105100	2
18	Retaining ring DIN471	1904807	1	43	M6x70 DIN6912	1902583	2
19	M10x35	1009442	2	44	Tube plug D28mm	1001606	2
20	Nut M10	1001683	3	45	P00042269	2658800	2
21	Washer M10 DIN127	1001566	2	46	X004580	1904798	2
22	X004123	1692600	1	47	P00042842	2995611	1
23	Pen D10x50	1998885	2	48	P00042843	2995610	1
24	M10x80 ISO7380	2992474	2	49	STICKER MAX. LOAD 130kg	2995632	2
				50	STICKER LOGO FALCON	T001194	2
				51	STICKER LOGO VERMEIREN	T001002	2
				52	M10 DIN439B	1903216	2
				Sheet 3/4			
				8003207			
				PATIENT HOIST FALCON			
				Plan No.			
				E00042257/J1.1			
				Scale: 1:12	Drafted	Verified	Confirmed
				Date	10/08/2022	22/06/2026	22/06/2026
				Initials	MK	PM	MK
				NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP. 			

Slings - Not depicted

No.	Plan	Article	Qty
	Comfort Net L	1905480	1
	Comfort Net M	1905479	1
	Comfort Net XL	1905481	1
	Comfort PE L	1905251	1
	Comfort PE M	1905359	1
	Comfort PE XL	1905360	1
	Hyg. Net L	1905483	1
	Hyg. Net M	1905482	1
	Hyg. Net XL	1905484	1
	Hyg. PE L	1905250	1
	Hyg. PE M	1905357	1
	Hyg. PE XL	1905358	1
	Reg. Net L	1905476	1
	Reg. Net M	1905475	1
	Reg. Net XL	1905477	1
	Reg. PE L	1903823	1
	Reg. PE M	1903824	1
	Reg. PE XL	1905478	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 4/4 8003207

PATIENT HOIST FALCON				Plan No.
				E00042257/J1.1
Scale: 1:12	Drafted	Verified	Confirmed	
Date	10/08/2022	22/06/2026	22/06/2026	
Initials	MK	PM	MK	