


No.	Plan	Article	Qty	26	Cap D22	1990648	2
1	E00062353	2997976	1	27	M6x12 ISO 7380	3992768	2
2	P00052625	2673765	1	28	P00053714	GP29100	1
3	P00052669	2674165	1	29	M8X75 DIN912 BLACK	2994374	2
4	P00052619	2673565	1				
5	P00053330	2673465	1	30	P00069216	2998122	4

6	P00052635 DP	2673865	1
7	P00053331	2673365	1
7*	P00119862	2923065	1
8	P00053221	2673265	1
8*	P00119877	2923165	1
9	X000106	G003218	2
10	P00052627	2997827	4
11	P00052597	2997829	4
12	P00052799	2997828	4
13	P00053093	GP28700	2
14	P00052802	2674400	2
15	P00052801	26743	4
16	Nut low M12X1,25	1994605	4
17	---	2998002	4
18	---	2998121	4
19	Washer M8 DIN125B	2994914	10
20	M8X40 DIN912 BLACK	3991566	6
21	M8X30 ISO7380 BLACK	2995396	2
22	NUT M8 DIN985	1996213	2
23	Washer PA6 M8 DIN125	2994415	2


* - use with E00119881 front wheel with suspension

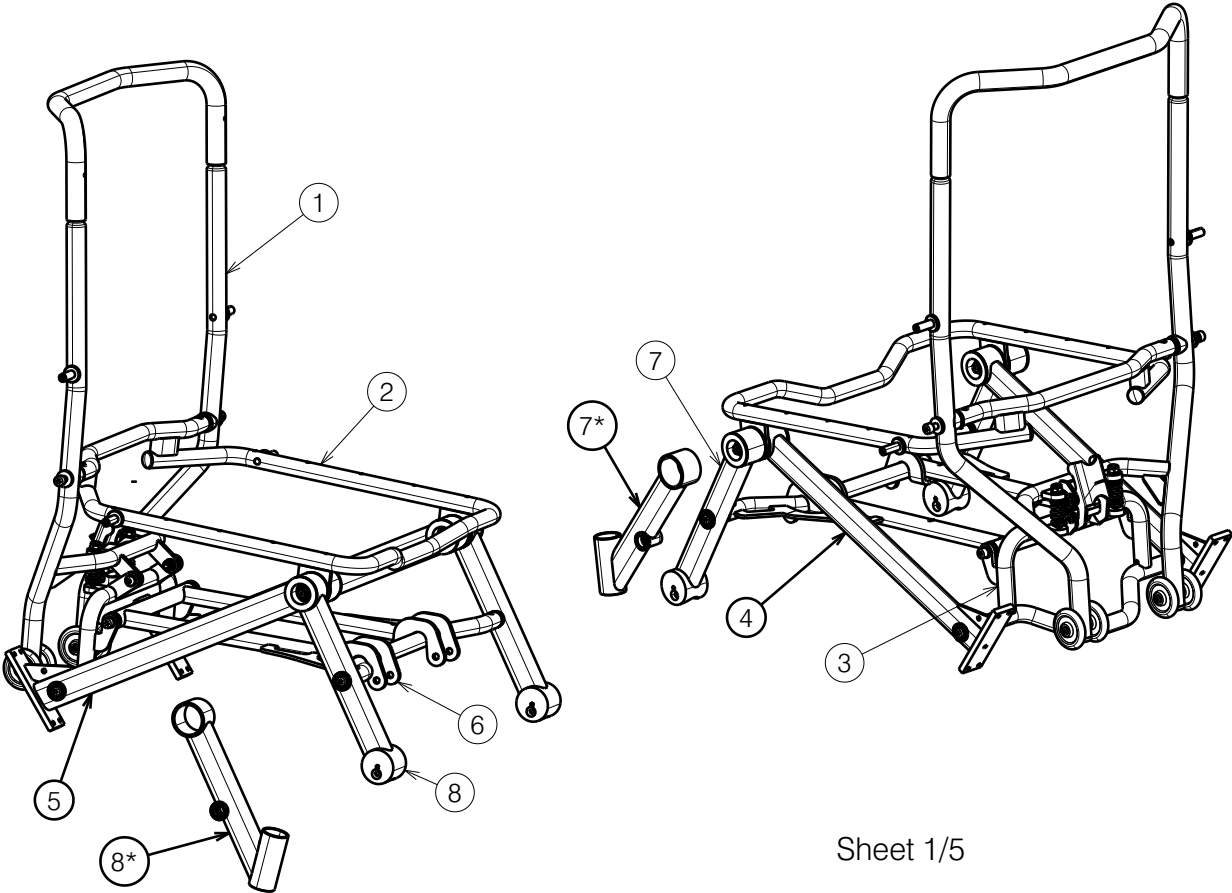

Sheet 1/5

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

BOM PLEGO FRAME				---
				Plan No. E00063648/K.1
Scale: 1:10	Drafted	Verified	Confirmed	
Date	27/09/2023	24/02/2026	24/02/2026	
Initials	BK	DR	AS	

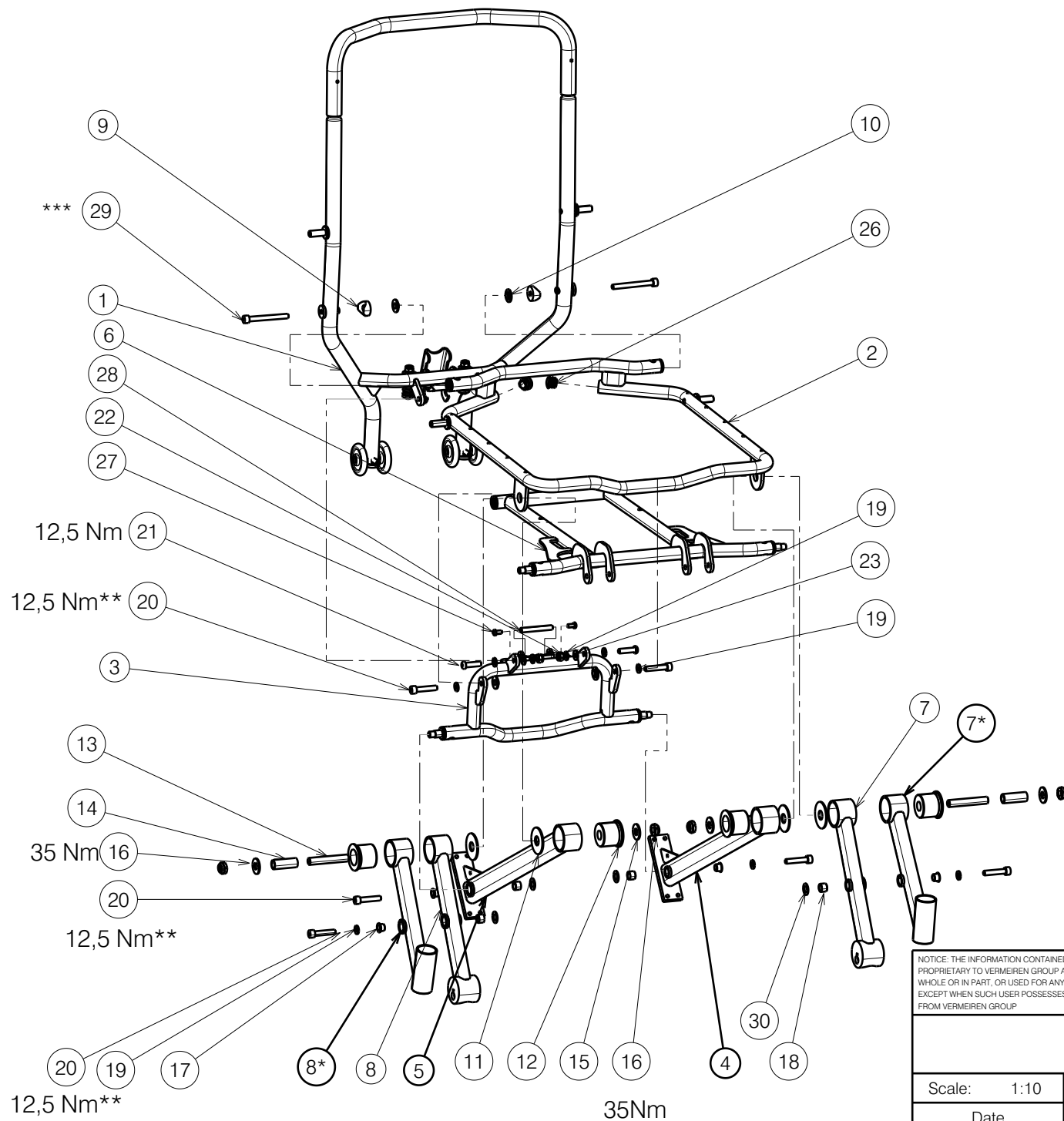



* Use with E00119881 front wheel with suspension
 ** Use WEICONLOCK AN 302-43
 *** Conditions of final assembly shown on drawing :
 E00052670

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

Sheet 2/5

BOM PLEGO FRAME			
Scale: 1:10	Drafted	Verified	Confirmed
Date	27/09/2023	24/02/2026	24/02/2026
Initials	BK	DR	AS

Plan No. E00063648/K.1

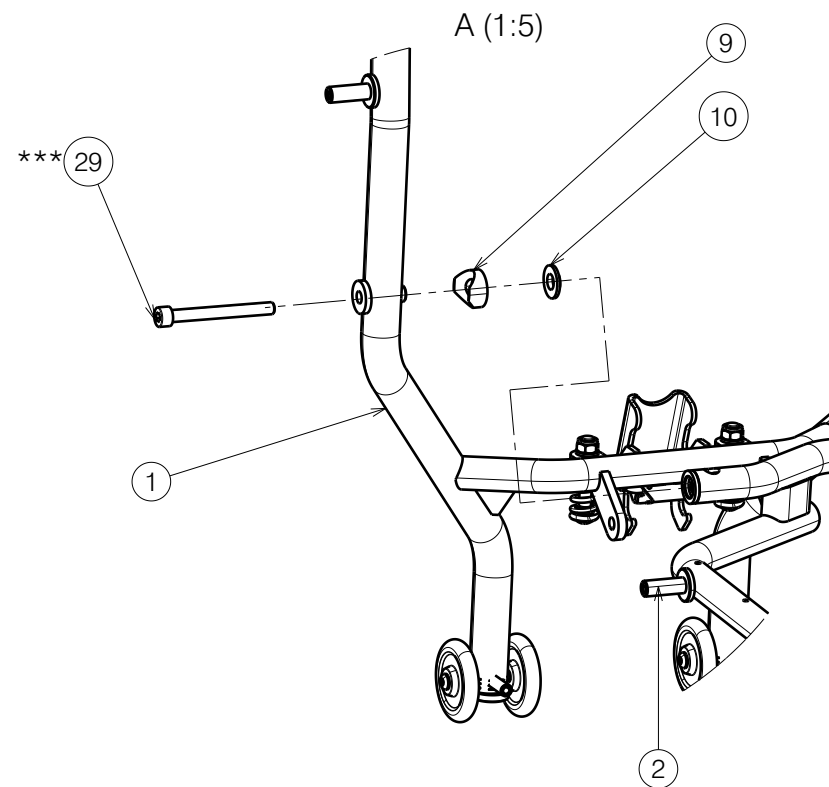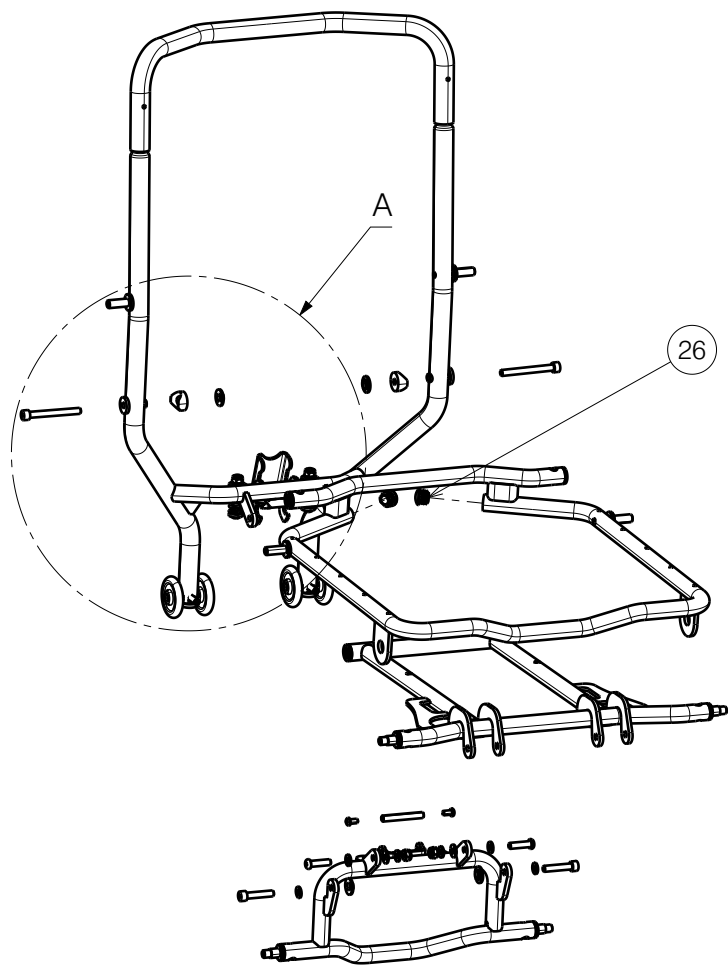

*** Conditions of final assembly shown on drawing :
E00052670

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 3/5


BOM PLEGO FRAME

Plan No.

E00063648/K.1

Scale: 1:10	Drafted	Verified	Confirmed
Date	27/09/2023	24/02/2026	24/02/2026
Initials	BK	DR	AS



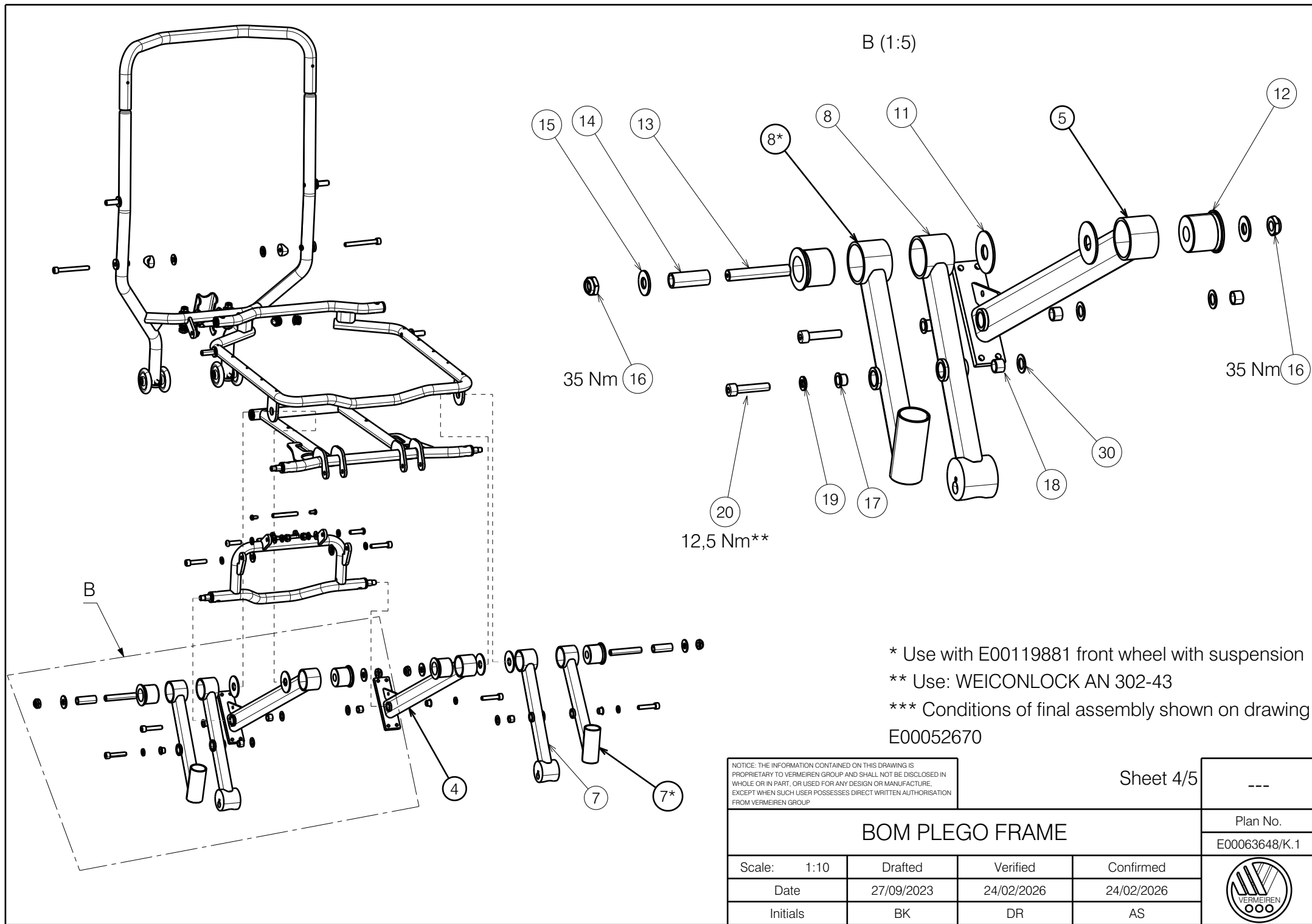

B (1:5)

35 Nm 16

12,5 Nm** 20

35 Nm 16

* Use with E00119881 front wheel with suspension

** Use: WEICONLOCK AN 302-43

*** Conditions of final assembly shown on drawing

E00052670

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 4/5


BOM PLEGO FRAME

Plan No.

E00063648/K.1

Scale: 1:10	Drafted	Verified	Confirmed
Date	27/09/2023	24/02/2026	24/02/2026
Initials	BK	DR	AS



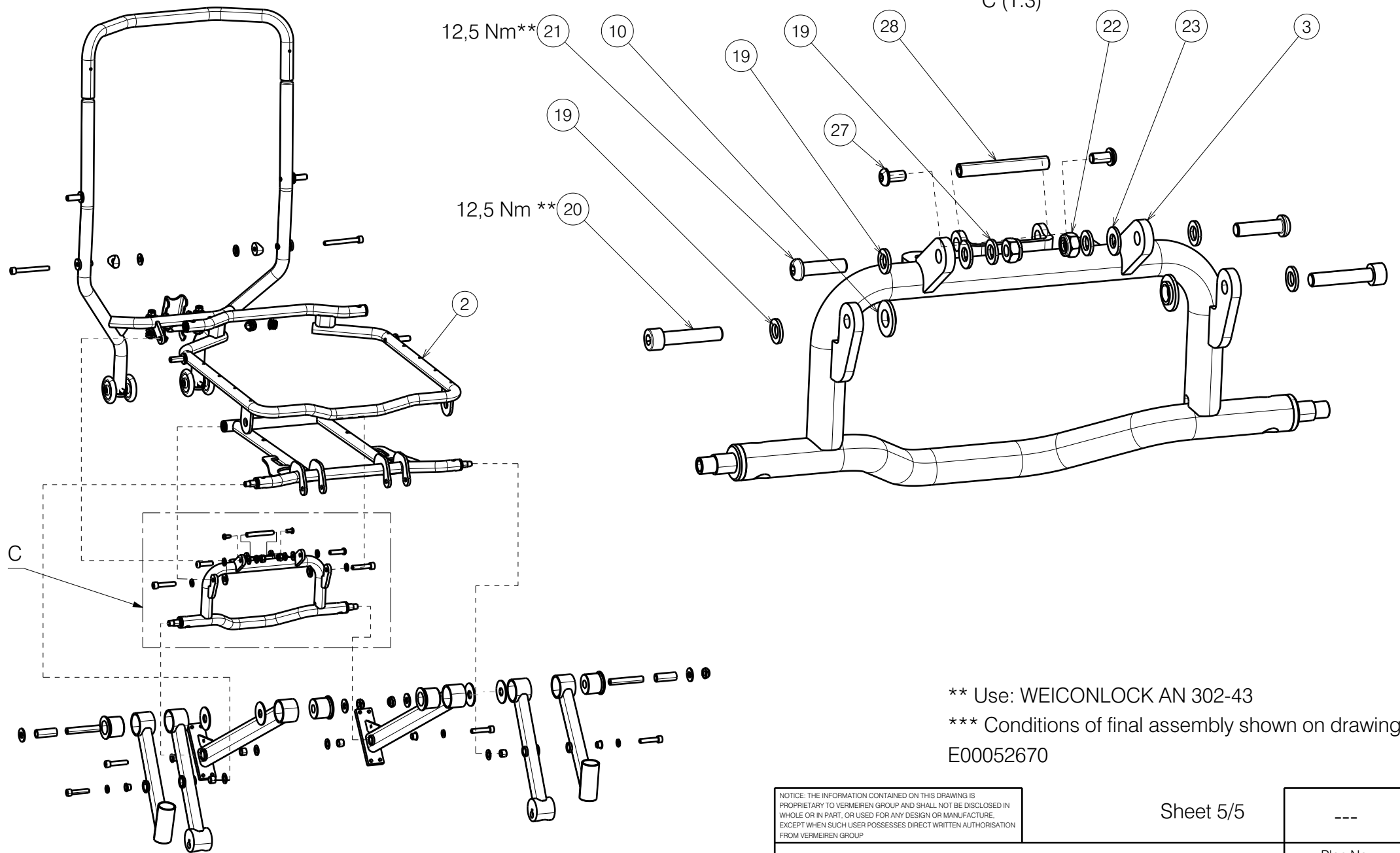

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 5/5

BOM PLEGO FRAME

Plan No.

E00063648/K.1

Scale: 1:10	Drafted	Verified	Confirmed
Date	27/09/2023	24/02/2026	24/02/2026
Initials	BK	DR	AS

