



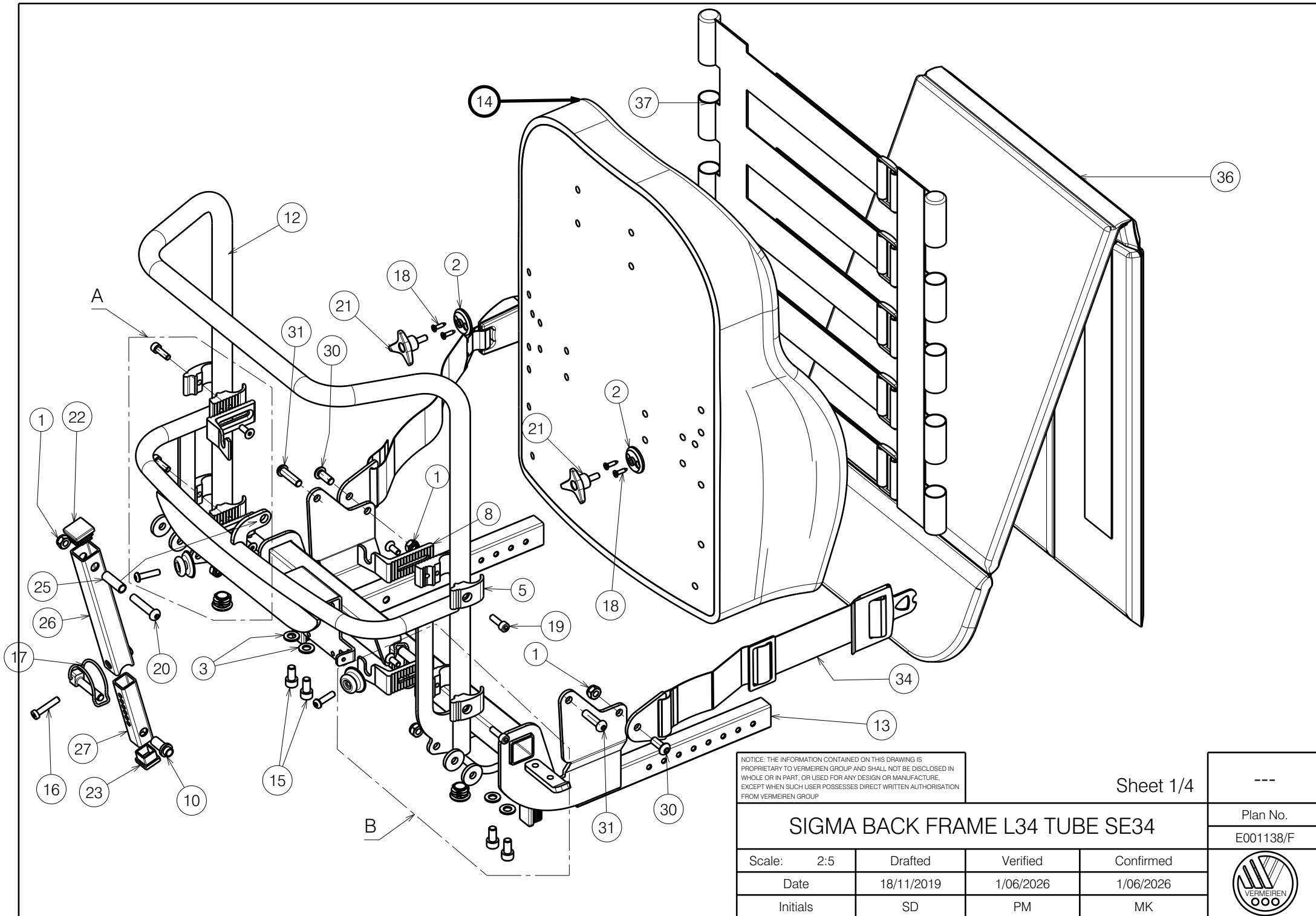


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/4

SIGMA BACK FRAME L34 TUBE SE34				Plan No.
Scale: 2:5				E001138/F
Date	18/11/2019	Verified	1/06/2026	
Initials	SD	PM	MK	

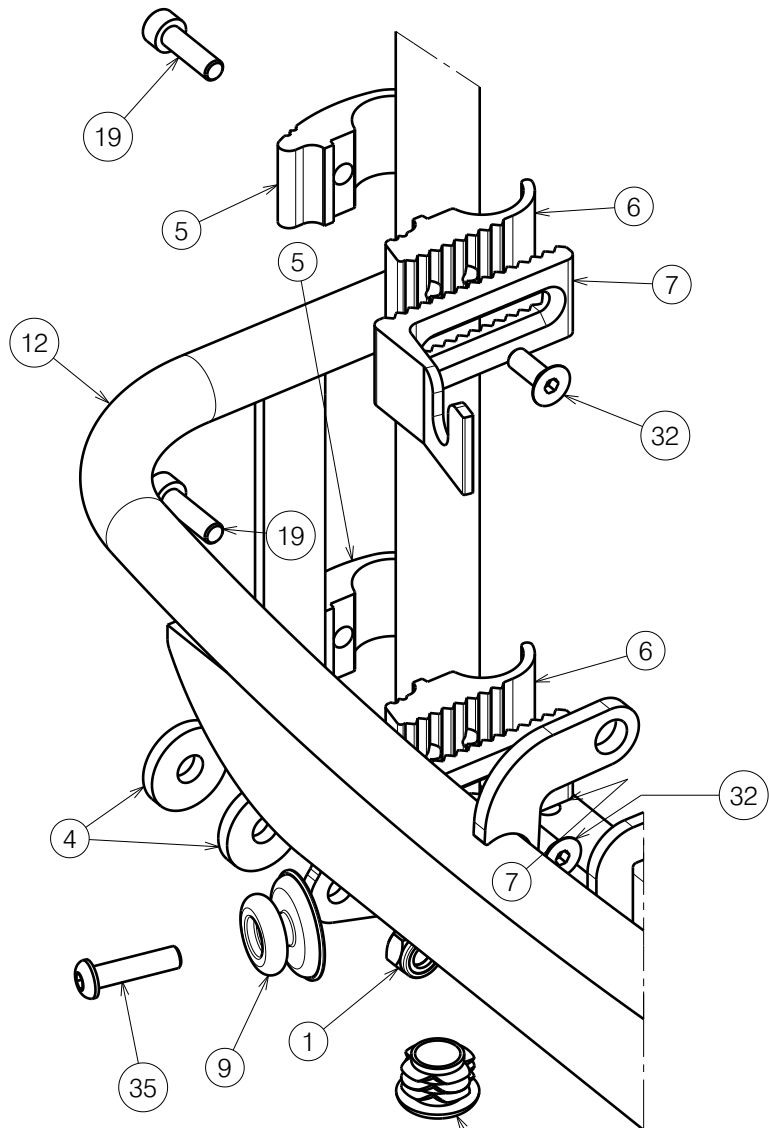

Detail A
Scale: 1:2

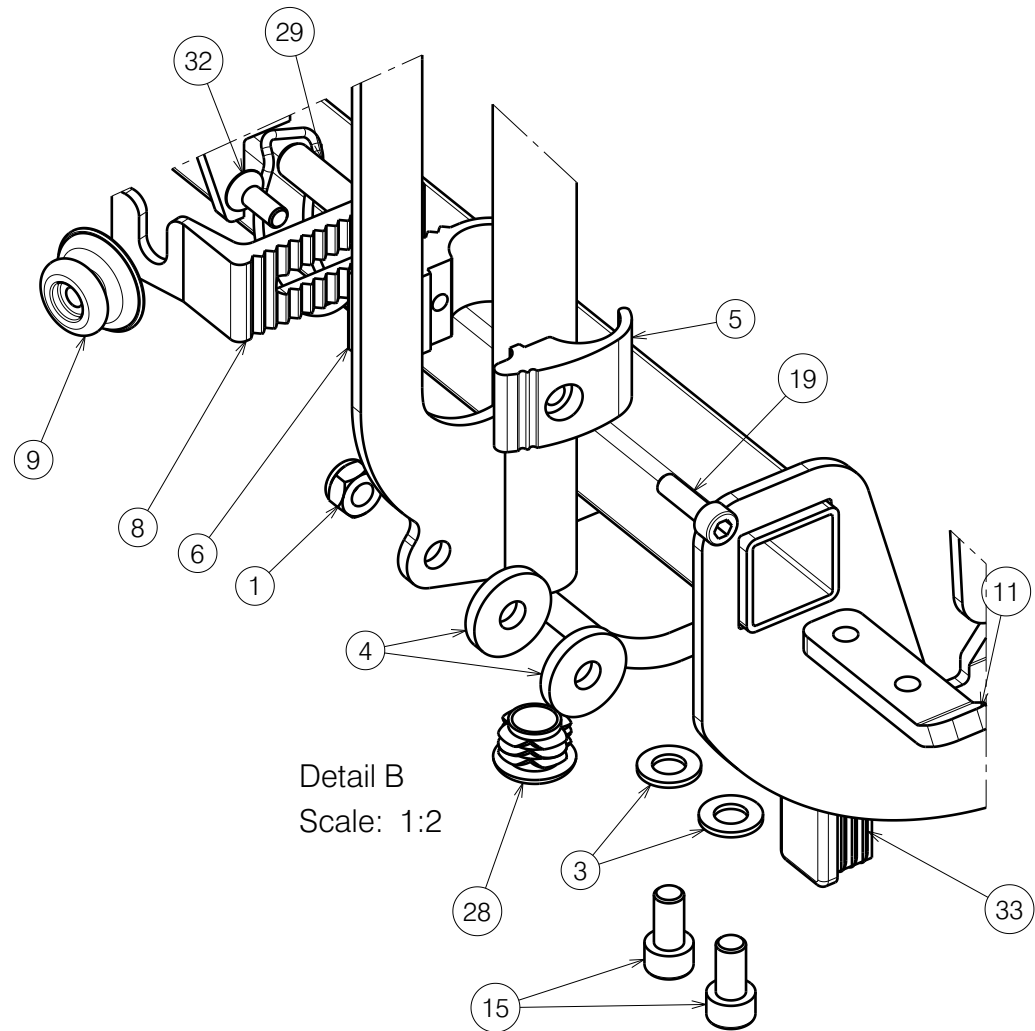

Detail B
Scale: 1:2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 2/4

SIGMA BACK FRAME L34 TUBE SE34

Plan No.

E001138/F

Scale: 2:5	Drafted	Verified	Confirmed
Date	18/11/2019	1/06/2026	1/06/2026
Initials	SD	PM	MK



Bill of Material				Bill of Material				Material			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	X001016	1001682	4	21	M6x15	1009932	2	37	A00082037	1906843(40)	1
2	X001052	G066600	2	22	25x25	1001617	1		---	2998461(42)	1
3	X001119	1001554	4	23	20x20	1001612	1		---	1906844(45)	1
4	X004048	1902685	4	25	Y021512	2011700	1		---	2998462(48)	1
5	X004820	1744599	4	26	Y021526	2011554	1		---	1906845(50)	1
6	X004821	1744699	4	27	Y021513	2011454	1				
7	X004822	1744799	2	28	Ø22	1990648	2				
8	X004823	1744899	2	29	Ø10x40	1991481	1				
9	X004861	1906454	2	30	M8x20 ISO7380	1992349	2				
10	X005471	2011600	1	31	M8x30 ISO7380	1992717	2				
11	Y018564	1774200	2	32	M6x16 DIN7991	1994752	4				
12	YP09205	2638854 (40)	1	33	30x20	1997922	2				
	P00078007	2686754 (42)	1	34	---	2993352	1				
	YP09206	2638954 (45)	1	35	M6x25 ISO7380	1006998	2				
	P00078470	2686854 (48)	1	36	P00082952	2998324 (40)	1				
	YP09207	2639054 (50)	1		P00083522	2998446 (42)	1				
13	Y022312	2049854 (40)	1		X004857	1906740 (45)	1				
	P00077920	2686354 (42)	1		P00082953	2998447 (48)	1				
	Y022980	2049754 (45)	1		X004889	1906841 (51)	1				
	P00077989	2686454 (48)	1		Airflow	2998325 (40)	1				
	Y022313	2049954 (50)	1		Airflow	2998448 (42)	1				
14	Soft grey 40	1912001	1		Airflow	2995211 (45)	1				
	Soft grey 42	2998402	1		Airflow	2998449 (48)	1				
	Soft grey 45	1911493	1		Airflow	2995212 (51)	1				
	Soft grey 48	2998407	1		Cordura	2998326 (40)	1				
	Soft grey 50	1924046	1		Cordura	2998450 (42)	1				
	Soft black 40	2993418	1		Cordura	2995208 (45)	1				
	Soft black 42	2998408	1		Cordura	2998451 (48)	1				
	Soft black 45	2993419	1		Cordura	2995209 (50)	1				
	Soft black 48	2998409	1								
	Soft black 50	2993420	1								
15	M8x16 DIN912	1001498	4								
16	M6x30 DIN7984	1992721	1								
17	Ø8x40	1992510	1								
18	D3,9x16 DIN7982	1005767	4								
19	M6x20 DIN912	1001493	4								
20	M8x40 ISO7380	1008739	1								

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 3/4

Plan No.
E001138/F

Scale: 2:5

Drafted

Verified

Confirmed

Date 18/11/2019

1/06/2026

1/06/2026

Initials SD


PM

MK



SIGMA BACK FRAME L34 TUBE SE34

*Option.

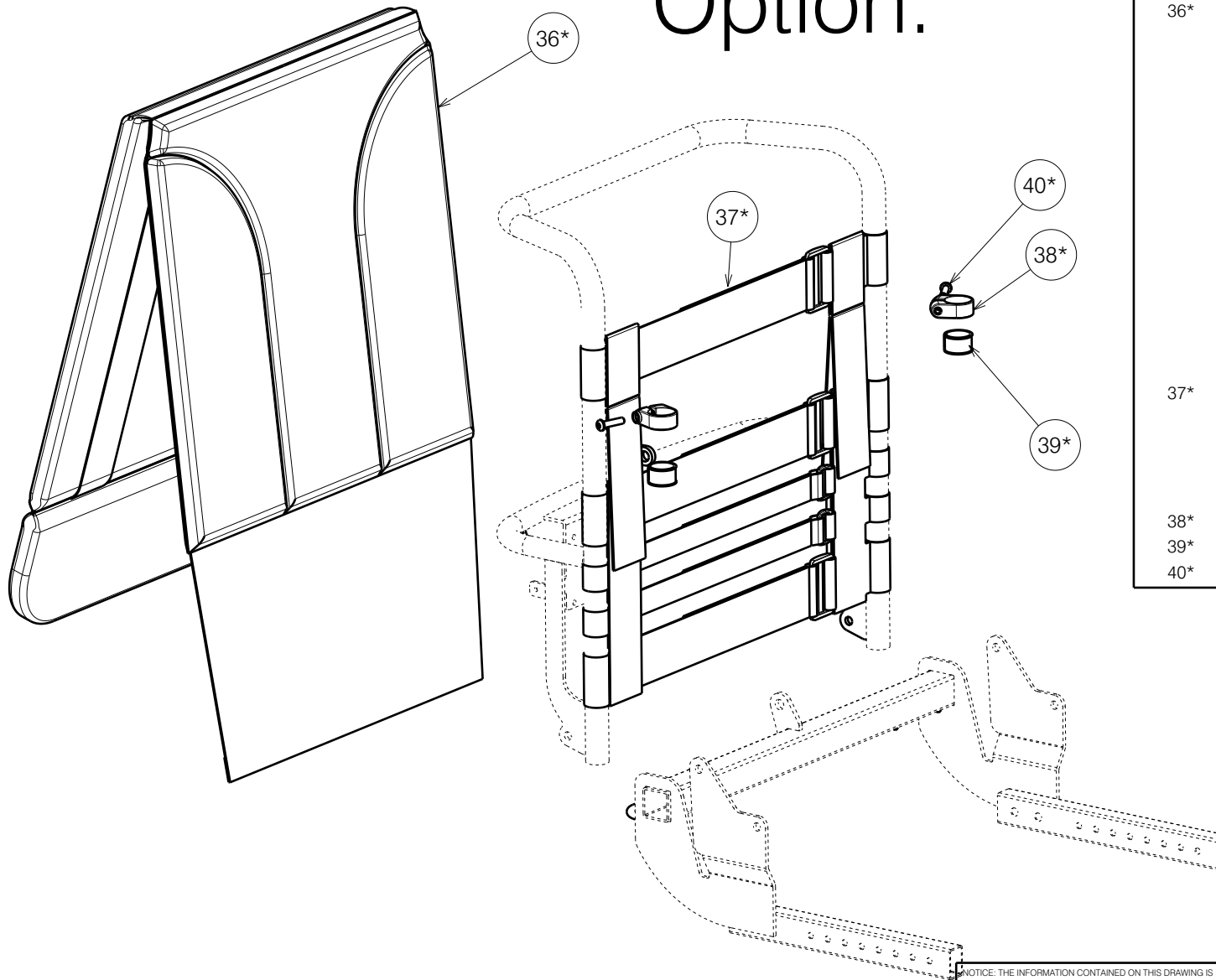

L44 WITH HEIGHT ADJUSTMENT			
No.	Plan	Article	Quantity
36*	P00079403	2998410(40)	1
	P00079237	2998411(42)	1
	P00079573	2998412(45)	1
	P00079238	2998413(48)	1
	P00079239	2998414(51)	1
	Airflow	2998467 (40)	1
	Airflow	2998426 (42)	1
	Airflow	2998427 (45)	1
	Airflow	2998428 (48)	1
	Airflow	2998429 (51)	1
	Cordura	2998468 (40)	1
	Cordura	2998432 (42)	1
	Cordura	2998433 (45)	1
	Cordura	2998434 (48)	1
	Cordura	2998435 (51)	1
37*	A00079244	2998415(40)	1
	---	2998416(42)	1
	---	2998417(45)	1
	---	2998418(48)	1
	---	2998419(51)	1
38*	XP01976	2993641	2
39*	XP01980	2993813	2
40*	M6x25	2994062	2

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 4/4

SIGMA BACK FRAME L34 TUBE SE34

Plan No.

E001138/F

Scale: 2:5	Drafted	Verified	Confirmed
Date	18/11/2019	1/06/2026	1/06/2026
Initials	SD	PM	MK

