
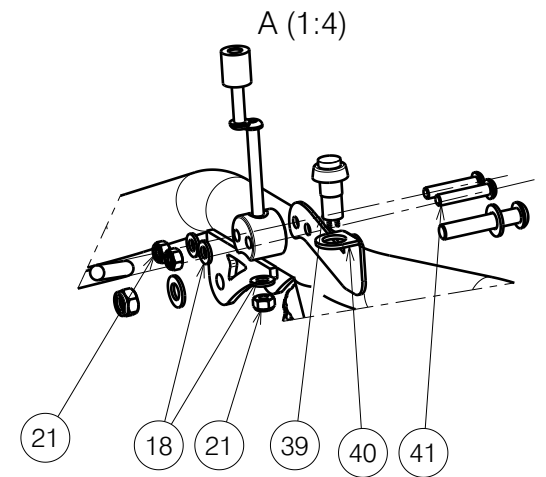


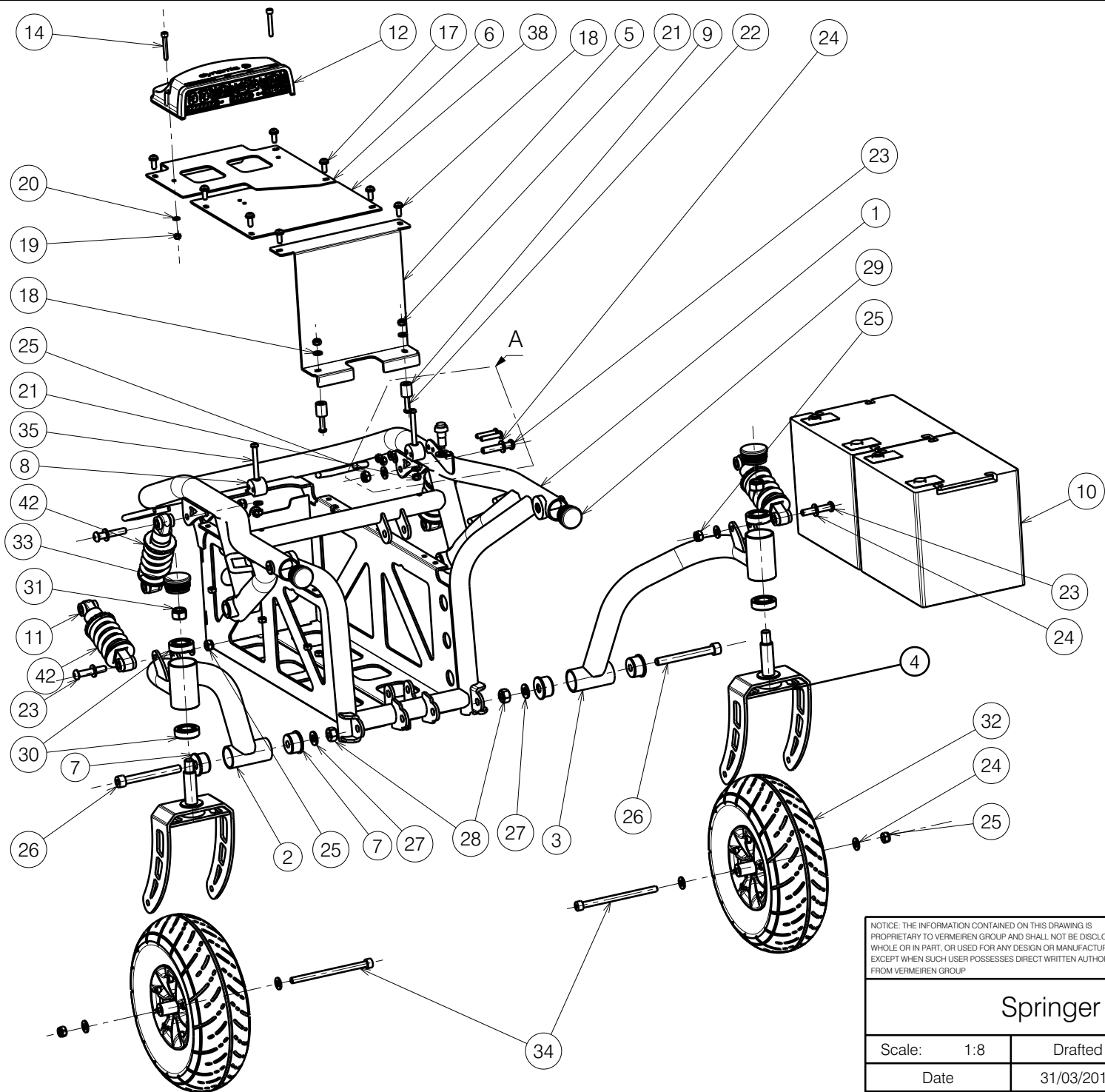

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

Springer Kids Main Frame			
Scale: 1:8	Drafted	Verified	Confirmed
Date	31/03/2011	23/06/2026	23/06/2026
Initials	AS	KA	AZ

Plan No.
EP00061/J


No.	Plan	Article	Qty
1	YP02258	18953..	1
2	YP02198	18951..	1
3	YP02200	18952..	1
4	YP02972	2819254	2
5	YP02752	2854854	1
6	YP02751	2854754	1
7	XP00562	2990709	4
8	XP00792	2991131	2
9	XP00795	2991286	2
10	Battery	1997015	2
11	---	1999843	4
12	LE 50A	1910896	1
13	-	-	-
14	DIN912-M5x35	1007961	2
15	-	-	-
16	-	-	-
17	ISO7380-M6x16	1008459	8
18	DIN125A-M6	1000973	14
19	DIN985-M5	1001679	2
20	M5	1000972	2
21	DIN985-M6	1001680	6

22	ISO7380-M6x35	1006057	2
23	ISO7380-M8x45	1994473	4
24	X001119	1001554	12
25	DIN985-M8	1001682	6
26	DIN912-M10x90	2992421	2
27	DIN125A-M10	1001558	2
28	DIN985-M10	1001683	2
29	---	1001610	2
30	---	1001409	4
31	DIN985-M12	1001684	2
32	Block grey	1915158	2
	Block black	1915159	2
	Line black L	1915160	1
	Line black R	1915161	1
	Line bl. L Rolko-fill	1915162	1
	Line bl. R Rolko-fill	1915163	1
	Block bl. Rolko-fill	1915164	2
33	---	1001611	2
34	DIN912-M8x120	1994676	2
35	M6x50	1006997	2
36	-	-	-
37	-	-	-
38	YP03009	28213	1
39	---	1903130	1
40	YP03011	2821754	1
41	ISO7380-M6x30	1009588	2
42	XP00863	2829154	4

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Springer Kids Main Frame				---
				Plan No.
				EP00061/J
Scale: 1:8	Drafted	Verified	Confirmed	
Date	31/03/2011	23/06/2026	23/06/2026	
Initials	AS	KA	AZ	