


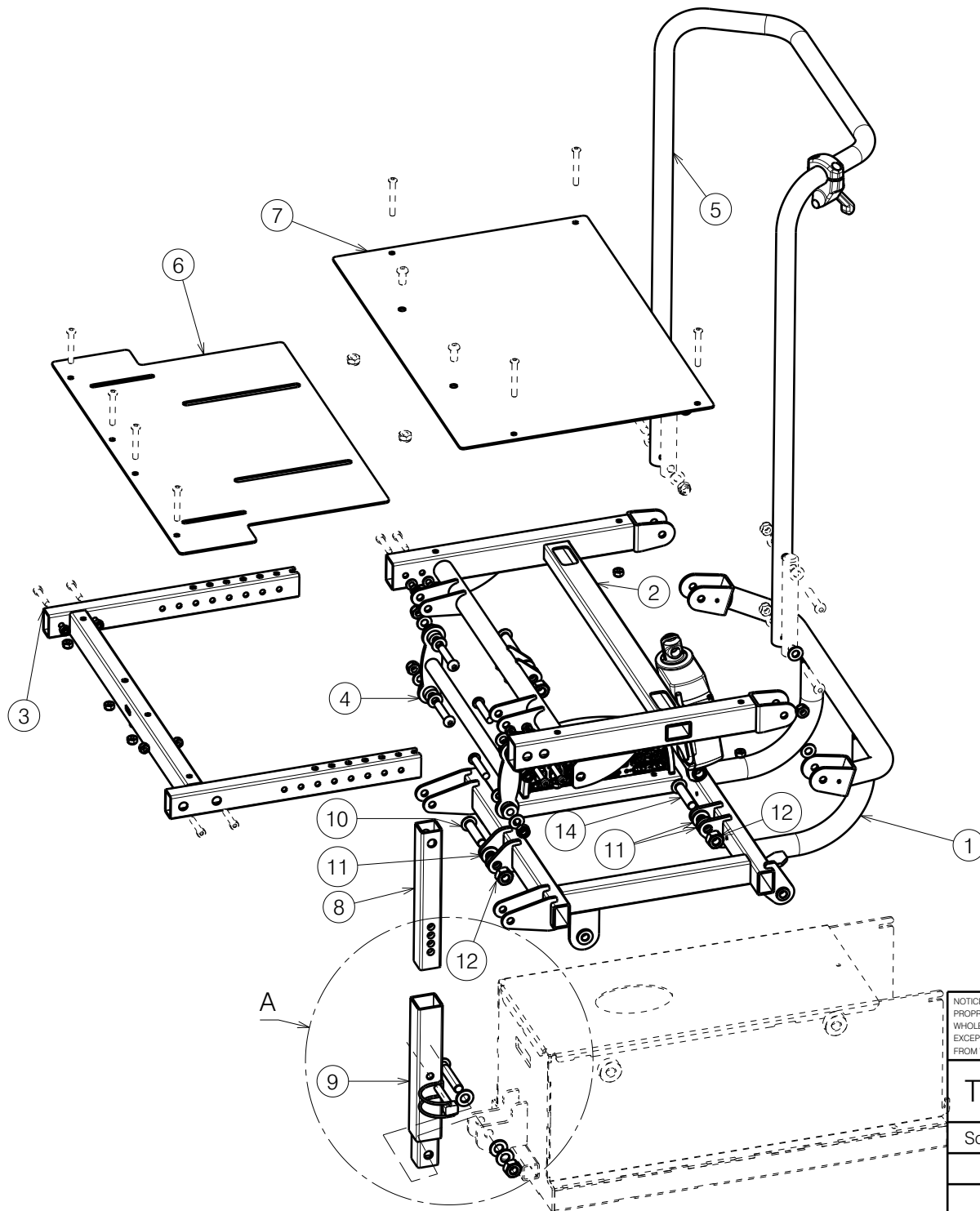


Seat adjustment angles:

A-a = 0deg

A-b = 2deg

A-c = 4deg

A-d = 6 deg

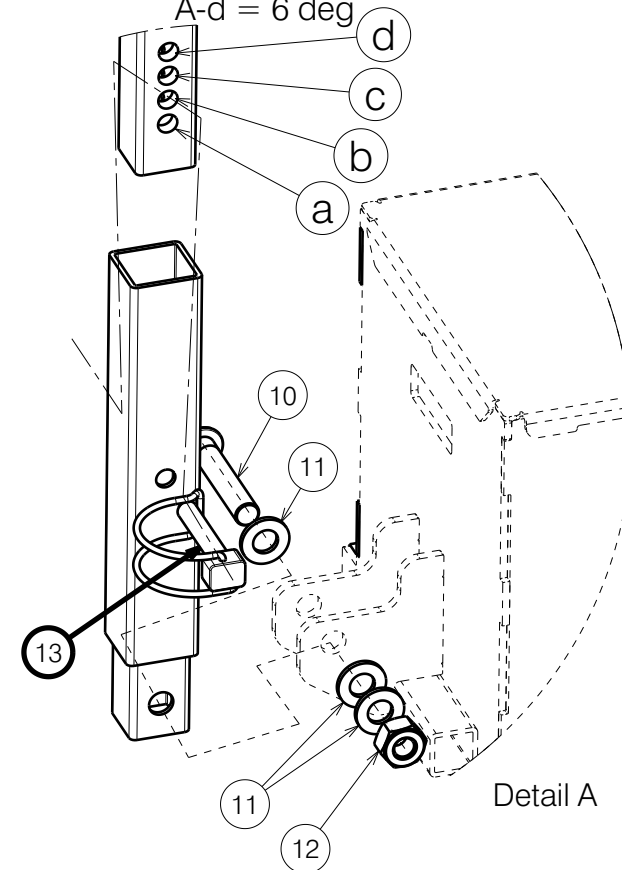

Detail A

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/2

TURIOS MAN SEAT SE30 AND EL BACK SE41

Scale:	1:7	Drafted	Verified	Confirmed
Date	2021-10-27	26/06/2026	26/06/2026	
Initials	MK	PM	MK	

Plan No.
EP00625/E1.1



Bill of Material			
No.	Plan	Article	Quantity
1	YP07411 (39)	2776954	1
	P00080268(42)	2689854	1
	YP07412 (45)	2777054	1
	P00080270(48)	2689954	1
2	YP07413 (51)	2777154	1
	YP07414 (39)	2777254	1
	P00080198(42)	2688954	1
	YP07415 (45)	2777354	1
3A	P00080218(48)	2689054	1
	YP07416 (51)	2777454	1
	YP07417 (39)	2777554	1
	P00080264(42)	2689154	1
3B*	YP07418 (45)	2777654	1
	P00080267(48)	2689254	1
	YP07419 (51)	2777754	1
	P00087337(39)	2692554	1
*	P00109052(42)	2910154	1
*	P00087866 (45)	2692854	1
*	P00106491(48)	2910254	1
*	P00088204 (51)	2692954	1
4	YP07421	2777954	1
5	YP07682 (39)	2600854	1
	P00080272(42)	2690454	1
	YP07683 (45)	2600954	1
	P00080293(48)	2690554	1
6	YP07684 (51)	2601054	1
	YP07706 (39)	2779754	1
	P00080301(42)	2690254	1
	YP07707 (45)	2779854	1
7	P00080303(48)	2690354	1
	YP07708 (51)	2779954	1
	YP07709 (39)	2780054	1
	P00080319(42)	2690054	1
8	YP07710 (45)	2780154	1
	P00080302(48)	2690154	1
	YP07711 (51)	2780254	1
	YP07736	27817	1

Bill of Material			
No.	Plan	Article	Quantity
9	YP09294	26433	1
10	M10x55	1998518	2
11	WASHER M10	1001558	10
12	NUT M10	1991858	4
13	M8x40	1993594	1
14	M10x45	1908848	2

* - Not depicted |

3A - Use with B06 |

3B - Use with BZ7/8 and BZE |

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/2

Plan No.

EP00625/E1.1

TURIOS MAN SEAT SE30 AND EL BACK SE41

Scale: 1:7

Drafted

Verified

Confirmed

Date

2021-10-27

26/06/2026

26/06/2026

Initials

MK

PM

MK