

No.	Plan	Article	Qty
1	YC01478	1C47304	1
2	A00093392*/ A00093393	1921941*/ 1921945	1
3	A00093396/ A00093395**	1921940**/ 1921944	1
4	E00082508	-	1
5	E00082509	-	1
7A	P00126183	2994482	1
7B	P00116147	GP32800	1
8	YC00613	1C22204	2
9	Screw M6x30 DIN912	1001495	1
10	Screw M10x30	1001502	4
11	Washer M6	1001564	1
12	Spring Washer M10 DIN127B	1001566	4
13	Nut M6	1001680	7
14	Nut M10	1001683	4
15	Screw M6x20	1006999	2
16	Cap Nut M3	2990306	2
17	Screw M6X8	3990062	5
18	Screw M3X8	3990797	4
20	Magnet	3990722	1
21	XC00190	3990617	1
22	XC00274	3990732	4
23	XC00296	3990672	1
24	XC00318	3991129	2
25	P00100883	2904200	1
26	Screw M5x16	2994067	6
27	Screw M5x30 BLACK	1903983	2

Tighten to a
torque 86Nm

Tightening break track to wheel

Put the magnet to hole with glue keep
the magnet at the same side



Electrical components presented on 2998640

Optional Equipment:

* - Rear Block Profile Tire Left

** - Rear Block Profile Tire Right

*** - 7B Brake

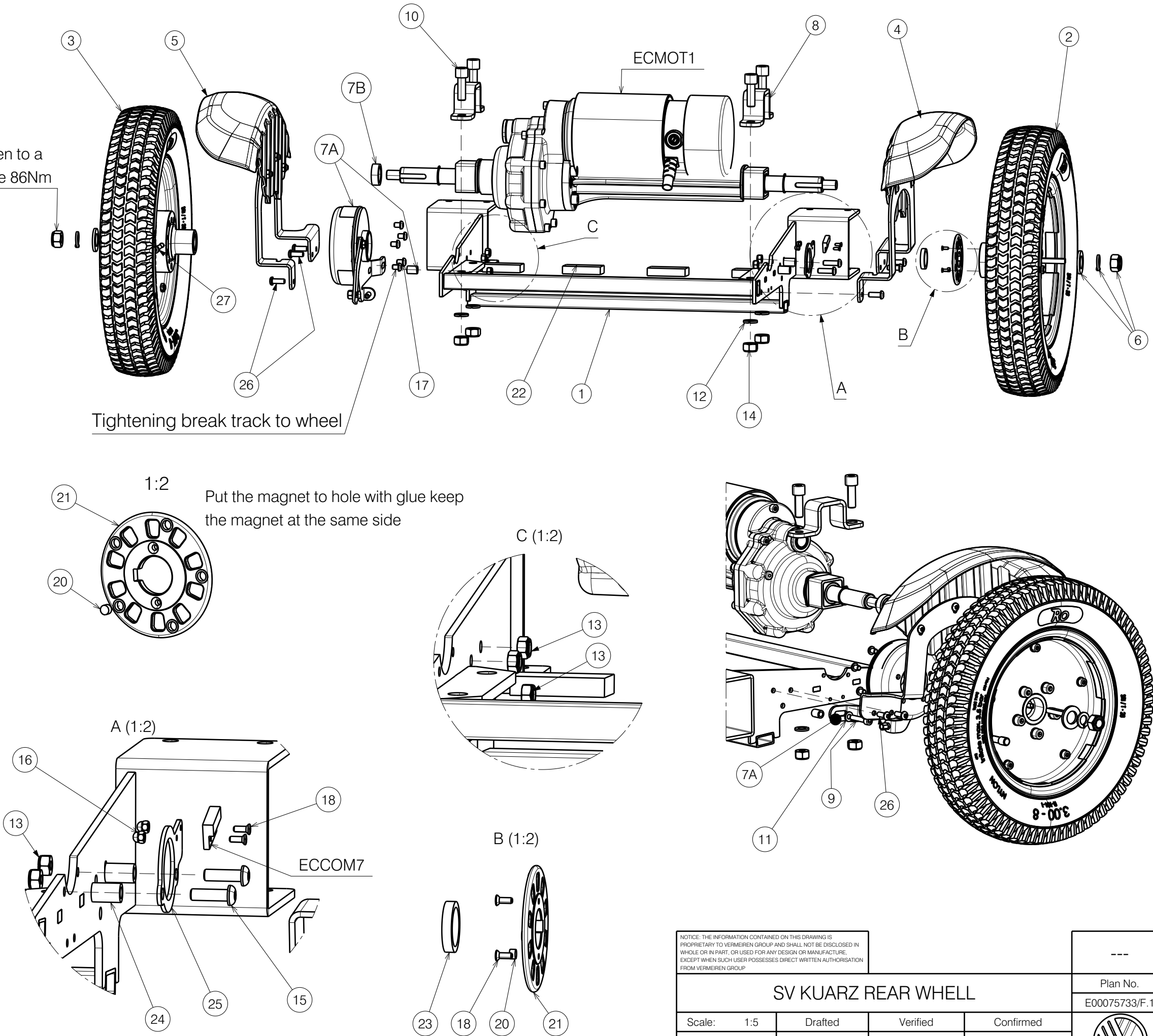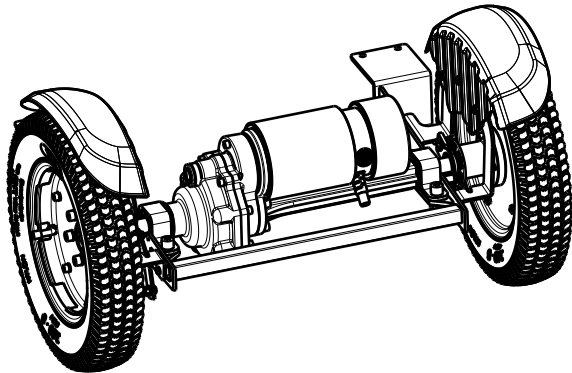

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION
FROM VERMEIREN GROUP

SV KUARZ REAR WHEEL

Scale:	1:5	Drafted	Verified	Confirmed
Date	04/12/2024	23/06/2026	23/06/2026	
Initials	BK	BK	MK	

Plan No. E00075733/F.1