


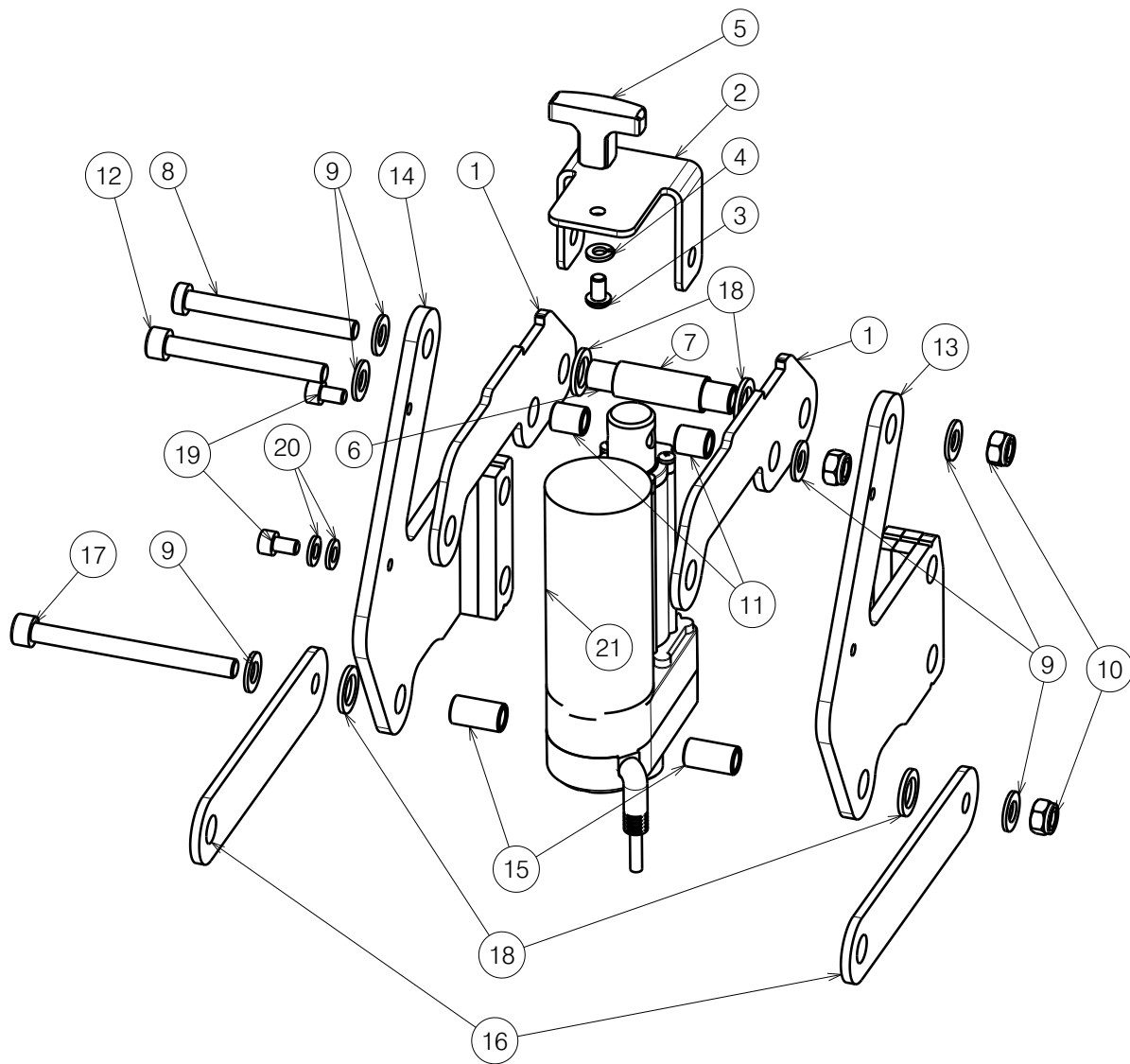

No.	Plan	Article	Qty
1	P00116709	2920254	2
2	P00120980	2921154	1
3	2994127BoltM6x10black	2994127	1
4	1004414SpringLockWasherDIN127M6	1004414	1
5	P00121226	2980011	1
6	P00118936	GP336	1
7	P00116672	GP331	1
8	2994912	2994912	1
9	WASHER M8 DIN126 BLACK ZINC	1996214	6
10	M8 DIN985 BLACK	1996213	3
11	P00115681	GP333	2
12	1005289InnerHexM8x70	1005289	1
13	P00118924 DP1	2919854	1
14	P00118924 DP2	2919954	1
15	P00116714	GP334	2
16	P00116717	2920354	2
17	1915949 M8x90	1915949	1
18	PVC M12 DIN433	2993281	4
19	M6X10 DIN912/8.8	1001491	2
20	DIN125A M6 PA6	1000973	2
21	MOTOR BLUM.SPORT 48VDC 80W	2980012	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

LIFT MECHANISM 1ST STEP BLUMIL SPORT

Scale: 1:5	Drafted	Verified	Confirmed
Date	26.05.2026	26.05.2026	26.05.2026
Initials	KK	PM	MK

Plan No.
E00127220/A.1


VERMEIREN GROUP