


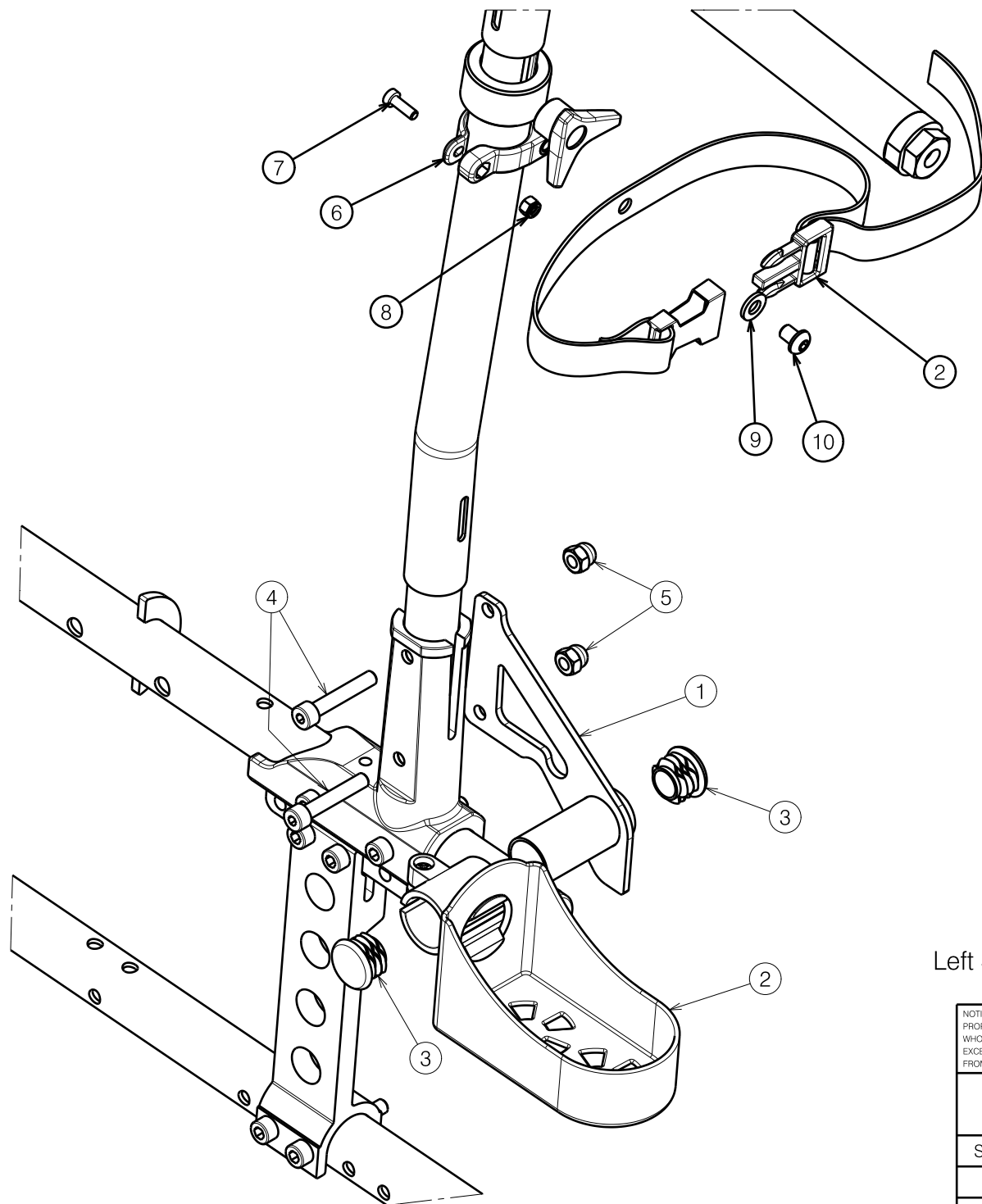

Bill of Material			
No.	Plan	Article	Quantity
1	P00077546	2091254 (L)	1
	P00078473	2091354 (R)	1
2	D22	1907511	1
3	D22	1990648	2
4	M6x35 DIN912	2994634	2
5	M6 DIN986	2994059	2
6	XP02043	2993671	1
7	M4x12 DIN7984	1905782	1
8	M4 DIN985	2994901	1
9	M6 DIN125	2994414	1
10	M6x8 ISO7380	2996034	1

Left Side Depiced - With BI5 Armrest

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

Sheet 1/2

VERSO B31 CRUTCH CARRIER

Plan No.

E00079628/C.1

Scale: 1:2

Drafted

Verified

Confirmed

Date

11/04/2024

08/07/2026

08/07/2026


Initials

SD

PA

PJ



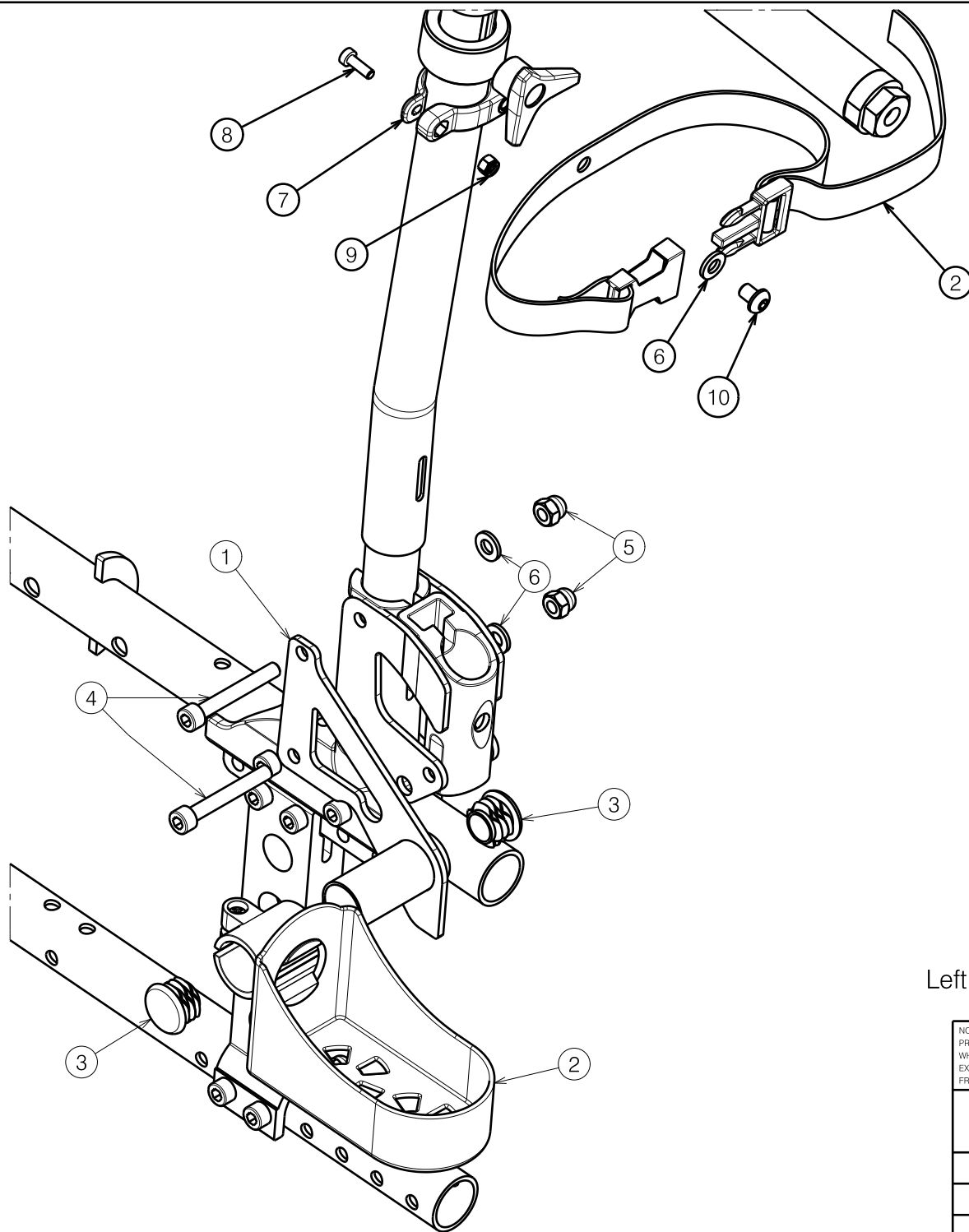

Bill of Material			
No.	Plan	Article	Quantity
1	P00078473	2091354 (L)	1
	P00077546	2091254 (R)	1
2	D22	1907511	1
3	D22	1990648	2
4	M6x45 DIN912	2994058	2
5	M6 DIN986	2994059	2
6	M6 DIN125	2994414	3
7	XP02043	2993671	1
8	M4x12 DIN7984	1905782	1
9	M4 DIN985	2994901	1
10	M6x8 ISO7380	2996034	1

Left Side Depiced - With B02/B03 Armrest

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 2/2

VERSO B31 CRUTCH CARRIER

Plan No.

E00079628/C.1

Scale: 1:2

Drafted

Verified

Confirmed

Date

11/04/2024

08/07/2026

08/07/2026

Initials

SD

PA

PJ

