



Tighten with torque wrench W600135 / 25 Nm: n°75 - n°39

Use loctite n°243 on n°59 - n°42

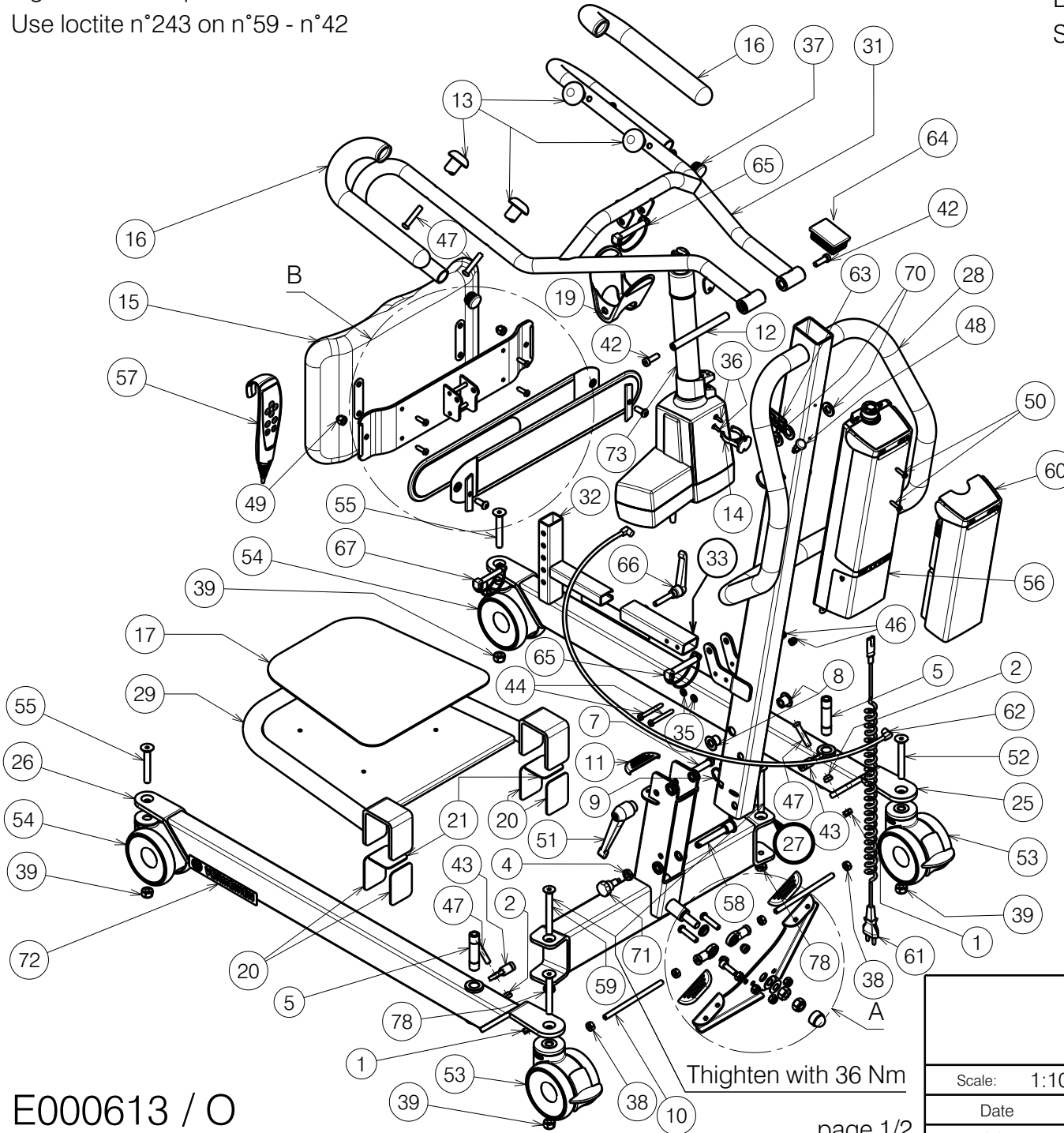


Detail A

Scale: 1:4

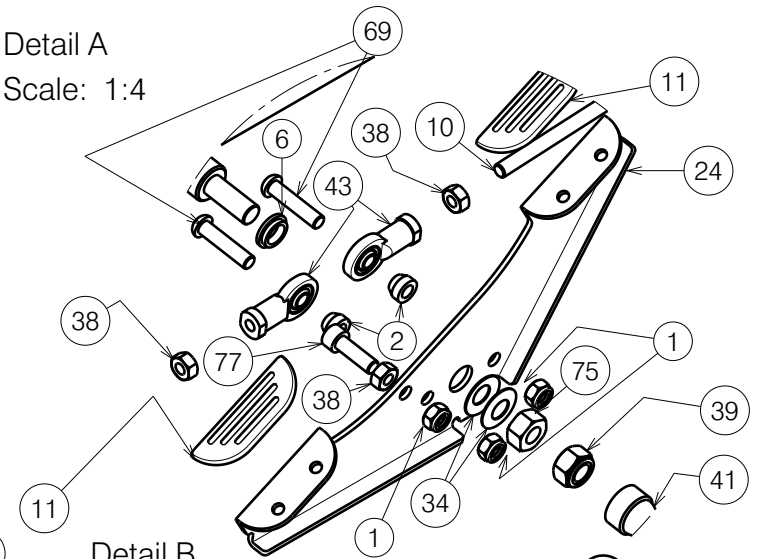

Detail B

Scale: 1:5

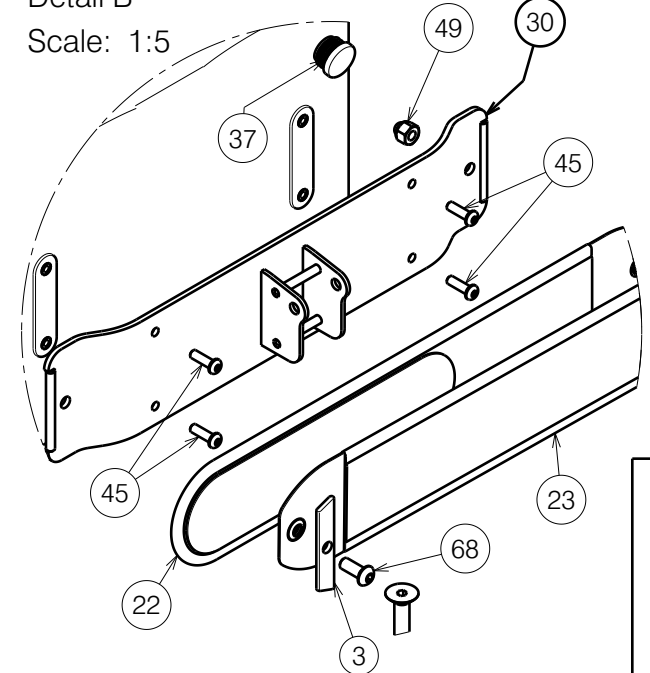

8001793

E000613 / O

Tighten with 36 Nm

page 1/2

ALBATROS

Plan No.

E000613/ O

Scale: 1:10

Drafted

Verified

Confirmed

Date

05/11/2010

02/10/2025

02/10/2025

Init.

TM

PM

MK



| Material |           |         |          |
|----------|-----------|---------|----------|
| No.      | Plan      | Article | Quantity |
| 1        | X001016   | 1001682 | 5        |
| 2        | P00074772 | 2098400 | 4        |
| 3        | Y017221   | 1743454 | 2        |
| 4        | X001398   | 1009689 | 1        |
| 5        | P00078139 | G105000 | 2        |
| 6        | X004288   | 1903587 | 1        |
| 7        | X004338   | 1708700 | 1        |
| 8        | X004341   | 1708800 | 2        |
| 9        | X004353   | 1903588 | 1        |
| 10       | P00075688 | 2098500 | 2        |
| 11       | X004580   | 1904798 | 3        |
| 12       | X004603   | 1734700 | 1        |
| 13       | X004605   | 1905610 | 4        |
| 14       | X004611   | 1904945 | 1        |
| 15       | X004630   | 1905252 | 1        |
| 16       | X004646   | 1905324 | 2        |
| 17       | X004647   | 1905718 | 1        |
| 19       | X004683   | 1905763 | 1        |
| 20       | X004704   | 1905720 | 4        |
| 21       | X004705   | 1905721 | 2        |
| 22       | X004717   | 1905722 | 1        |
| 23       | X004718   | 1905723 | 1        |
| 24       | P00074771 | 2098354 | 1        |
| 25       | Y014535   | 1706719 | 1        |
| 26       | Y014536   | 1706819 | 1        |
| 27       | Y014537   | 1706919 | 1        |
| 28       | P00039617 | 2082819 | 1        |
| 29       | Y016061   | 1735219 | 1        |
| 30       | Y016137   | 1732619 | 1        |

| Material |                |         |          |
|----------|----------------|---------|----------|
| No.      | Plan           | Article | Quantity |
| 31       | Y016174        | 1732319 | 1        |
| 32       | Y016385        | 1732519 | 1        |
| 33       | Y016465        | 1735319 | 1        |
| 34       | ---            | 1993500 | 2        |
| 35       | ---            | 1001564 | 2        |
| 36       | ---            | 1001584 | 2        |
| 37       | ---            | 1001608 | 2        |
| 38       | ---            | 1001671 | 5        |
| 39       | ---            | 1001683 | 4        |
| 40       | ---            | 1001684 | 1        |
| 41       | ---            | 1003294 | 1        |
| 42       | ---            | 1003944 | 2        |
| 43       | ---            | 1006835 | 4        |
| 44       | ---            | 1006997 | 2        |
| 45       | ---            | 1006999 | 4        |
| 46       | ---            | 1008492 | 2        |
| 47       | ---            | 1008739 | 6        |
| 48       | ---            | 1009583 | 1        |
| 49       | ---            | 1902024 | 2        |
| 50       | M5x20          | 1998368 | 2        |
| 51       | ---            | 1903086 | 1        |
| 52       | ---            | 1905617 | 2        |
| 53       | ---            | 1903402 | 2        |
| 54       | ---            | 1903403 | 2        |
| 55       | ---            | 1903406 | 2        |
| 56       | CAL40          | 1918030 | 1        |
| 57       | ---            | 1906138 | 1        |
| 58       | ---            | 1903578 | 1        |
| 59       | M10x90 DIN7991 | 1921197 | 2        |
| 60       | BAL40          | 1918029 | 1        |

| Material |             |         |          |
|----------|-------------|---------|----------|
| No.      | Plan        | Article | Quantity |
| 61       | ---         | 1906233 | 1        |
| 62       | ---         | 1918032 | 1        |
| 63       | ---         | 1904556 | 1        |
| 64       | ---         | 1905096 | 1        |
| 65       | ---         | 1905365 | 2        |
| 66       | ---         | 1905369 | 1        |
| 67       | ---         | 1905609 | 1        |
| 68       | ---         | 1992349 | 2        |
| 69       | ---         | 1993236 | 2        |
| 70       | ---         | 1993362 | 2        |
| 71       | ---         | 1999935 | 1        |
| 72       | 7.5 kN      | T001052 | 2        |
| 73       | ---         | 1918033 | 1        |
| 75       | ---         | 1001673 | 1        |
| 76*      |             | 1918034 | 1        |
| 77       | M8x25       | 1001499 | 1        |
| 78       | M10 DIN439B | 1903216 | 2        |

\* Not depicted - Optional ext. charger

8001793

E000613 / O

|             |            |            |            |                                                                                       |
|-------------|------------|------------|------------|---------------------------------------------------------------------------------------|
| ALBATROS    |            |            |            | Plan No.                                                                              |
|             |            |            |            | E000613/ O                                                                            |
| Scale: 1:10 | Drafted    | Verified   | Confirmed  |  |
| Date        | 05/11/2010 | 02/10/2025 | 02/10/2025 |                                                                                       |
| Init.       | TM         | PM         | MK         |                                                                                       |