


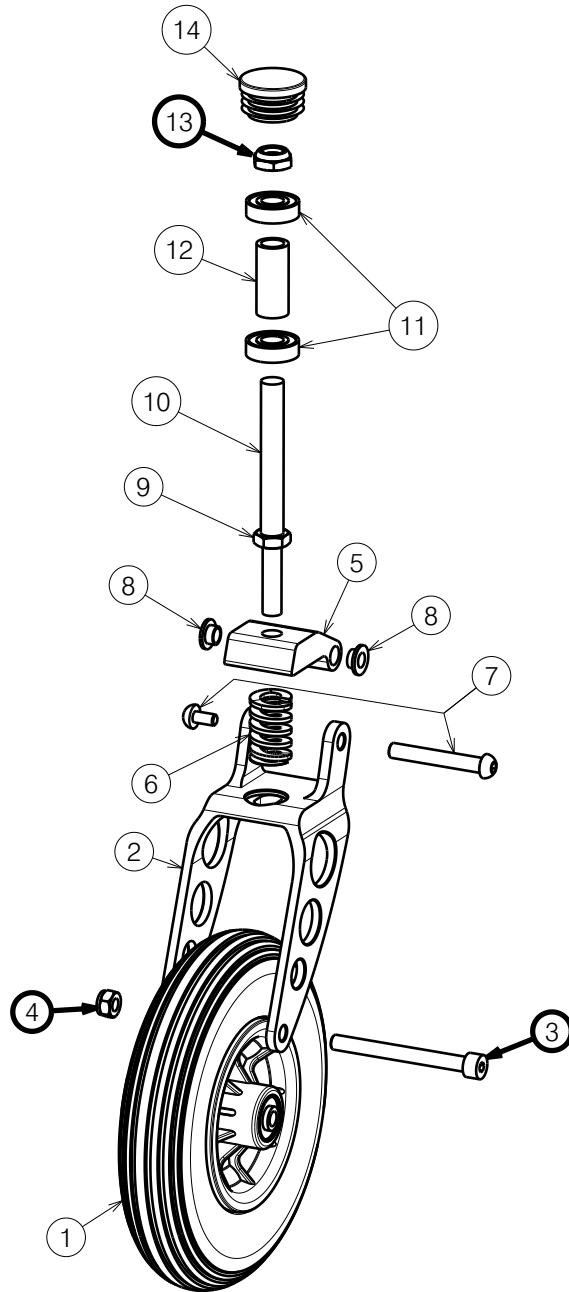

Bill of Material			
No.	Plan	Article	Quantity
1	Wheel 200x50 PU	1916559	1
2	Fork C56G	29225A7	1
	C4G	29225A9	1
	C87G	29225B8	1
	C88G	29225D5	1
	C52	2922521	1
	C75	2922565	1
	C77	2922576	1
3	M8x80 DIN912	2994121	1
4	Nut M8 lock	1996213	1
5	Block for fork C56G	29226A7	1
	C4G	29226A9	1
	C87G	29226B8	1
	C88G	29226D5	1
	C52	2922621	1
	C75	2922665	1
	C77	2922676	1
6	Spring	1923504	1
7	Axle	1923508	1
8	Bushing	1923506	2
9	Nut	1923505	1
10	Axle	1923507	1
11	Bearing 6001	1992559	2
12	P00119880	GP33700	1
13	Nut M12	1994605	1
14	Cap D35	2999421	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

PLEGO FRONT WHEEL WITH SUSPENSION

Scale: 1:4	Drafted	Verified	Confirmed
Date	20/02/2026	26/06/2026	26/06/2026
Initials	DR	AZ	AZ

-

Plan No.

E00119881/C.1

