



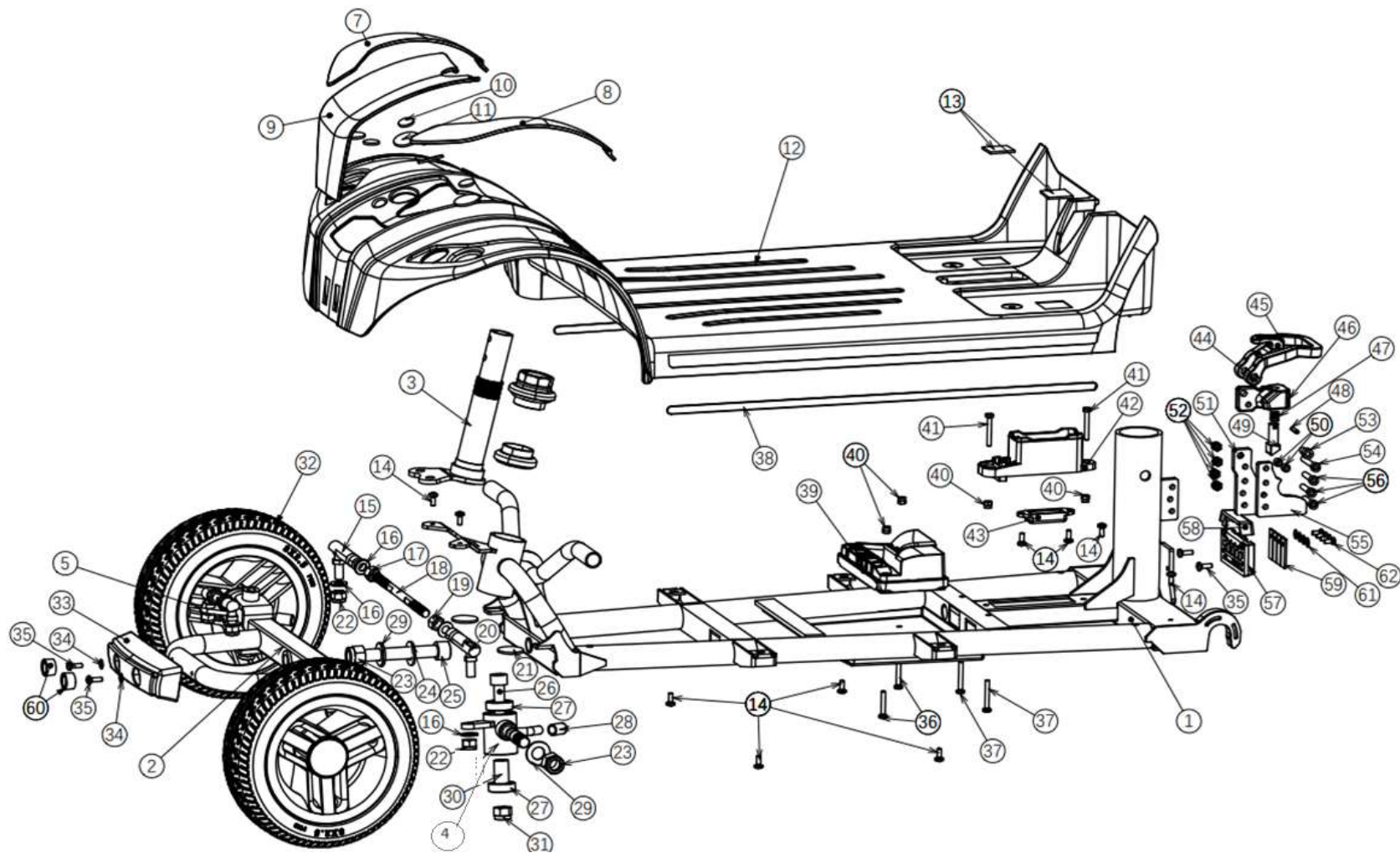


no	part nu	part	QT		
1	1920656	Main frame ONE	1		
2	1918862	front fork	1		
3	1918856	STEERING AXLE ONE	1		
4	1920339	FRONTAXLE L	1		
5	1918854	FRONTAXLE R	1		
6					
7	1920443	COLOR PANEL SET FRONT green			
7	1920444	COLOR PANEL SET FRONTorange			
7	1920445	COLOR PANEL SET FRONTbleu			
7	1920446	COLOR PANEL SET FRONT pink			
8					
8					
8					
8					
9	1918852	FRONTCOVER	1		
10					
11					
12	1918851	SHROUD ONE	1		
13					
14					
15	1918858	BALL JOINT RIGHT	2		
16					
17					
18	1918859	CONNECTION ROD	2		
19					
20	1918857	BALL JOINT LE			
21					
22					
23					
24					
25					
26					

27					
28					
29					
30					
31					
32	1918855	FRONTWHEEL ONE	2		
33	1920632	FRONT BUMPER	1		
34	1920634	WASHER M4 NF E25-513	6		
35	1920633	BOLT M4X20 DIN 7985			
36					
37					
38					
39	1919090	POWER MODULE	1		
40					
41					
42	1920657	BATTERY CONNECTOR HOLDER 1	1		
43	1920658	BATTERY CONNECTOR HOIDER 2	1		
	1922814	BATTERY CONNECTOR+CABLE	1		
44					
45					
46					
47					
48					
49					
50					
51					
52					
53					
54					
55					
56					
57	1918861	FRONT CONTACT POINT HOLDER	1		
58					


[illegible]

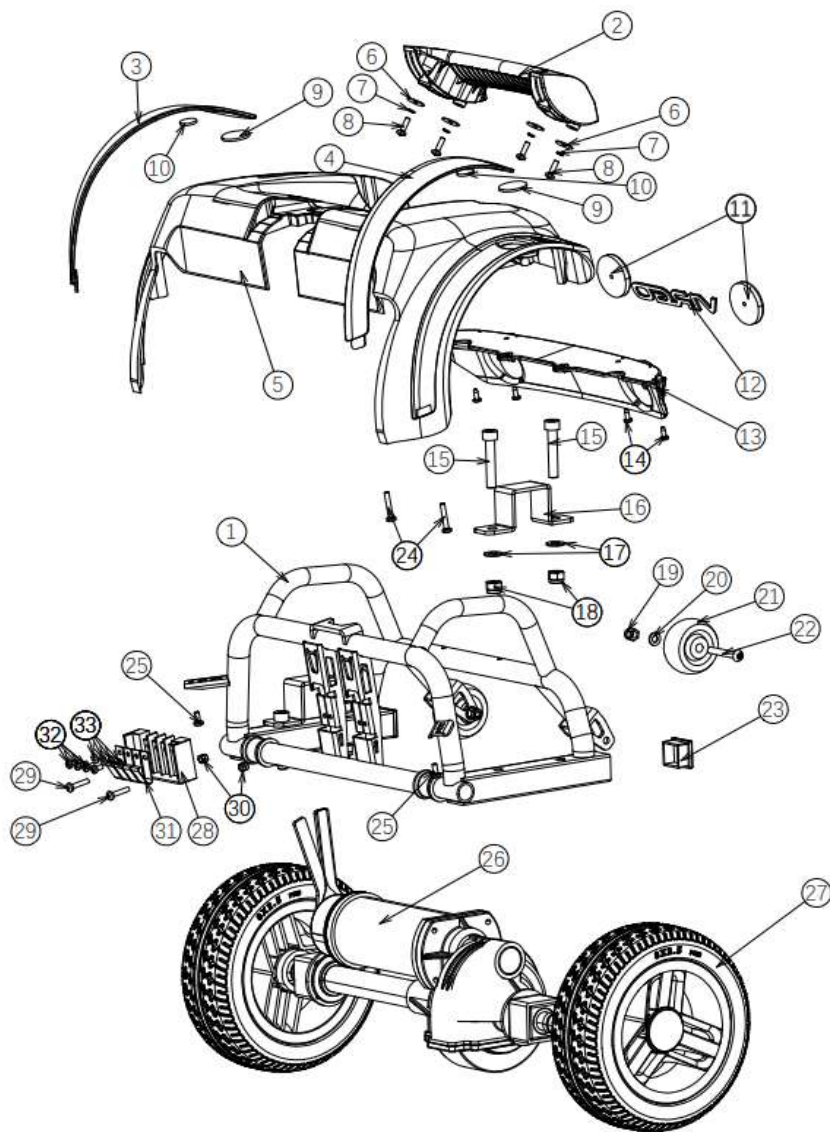


no	part nu	part	QT		part nu	part	QT
1				17			
2	1918886	HANDLE REAR	1	18			
3	1920439	COLOR PANEL SET REAR green		19	1922551	LOCK NUT	2
3	1920440	COLOR PANEL SET REAR orange		20	1922550	SPRING WASHER	2
3	1920441	COLOR PANEL SET REAR bleu		21	1922523	WHEEL ANTI TIPPING	2
3	1920442	COLOR PANEL SET REAR pink		22	1922549	BOLT ANTI TIP	2
4				23			
4				24			
4				25			
4				26	1918887	DIFF.+MOTOR+REM	1
				26	1920395	DIFF	1
				26	1920396	MOTOR	1
				NOT DEPICTED	1922014	BUSH TRANSMISION	1
				26	1920397	MOTOR BRAKE	1
5	1918853	COVER MIDDLE	1	27	1918888	REAR WHEEL	2
6	1922402	WASHER M4	4	28	1918889	REAR CONTACT POINT HOLDER	1
7	1922403	SPRING WASHER	4	29			
8	1922404	SCREW M4	4	30			
9				31	1918890	CONTACT POINT REAR	4
11	1921454	REAR RED REFLECTOR	2	32			
12	1922405	STICKER ONE	1	33			
13	1918882	REAR COVER	1				
14	1922406	SCREW ST3x9,5	4				
15							
16							
not depicted	1923143	AXLE KEY	2				
not depicted	1922190	RED MOTORBRAKE LEVER COVER	1				

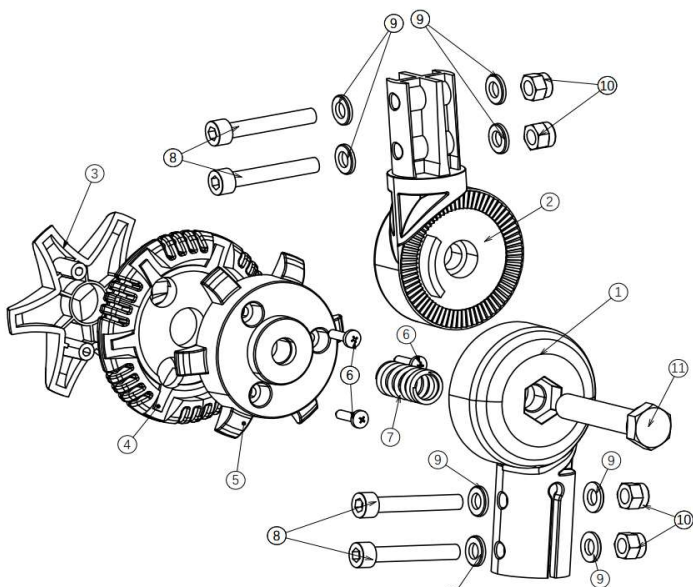


no	part nu	part	QT				
1	1921317	Lower part adjustment	1				
2							
3	1921320	knob angle adjust outside	1				
4	1921319	knob angel adjust middle	1				
5	1921318	knob angle adjust inside	1				
6	1921322	bolt M5 X 16	3				
7	1921321	spring angle adjust knob	1				
8							
9							
10							
11	1922522	Bolt M10x55	1				
not depicted	1922514	set 1/12	0				

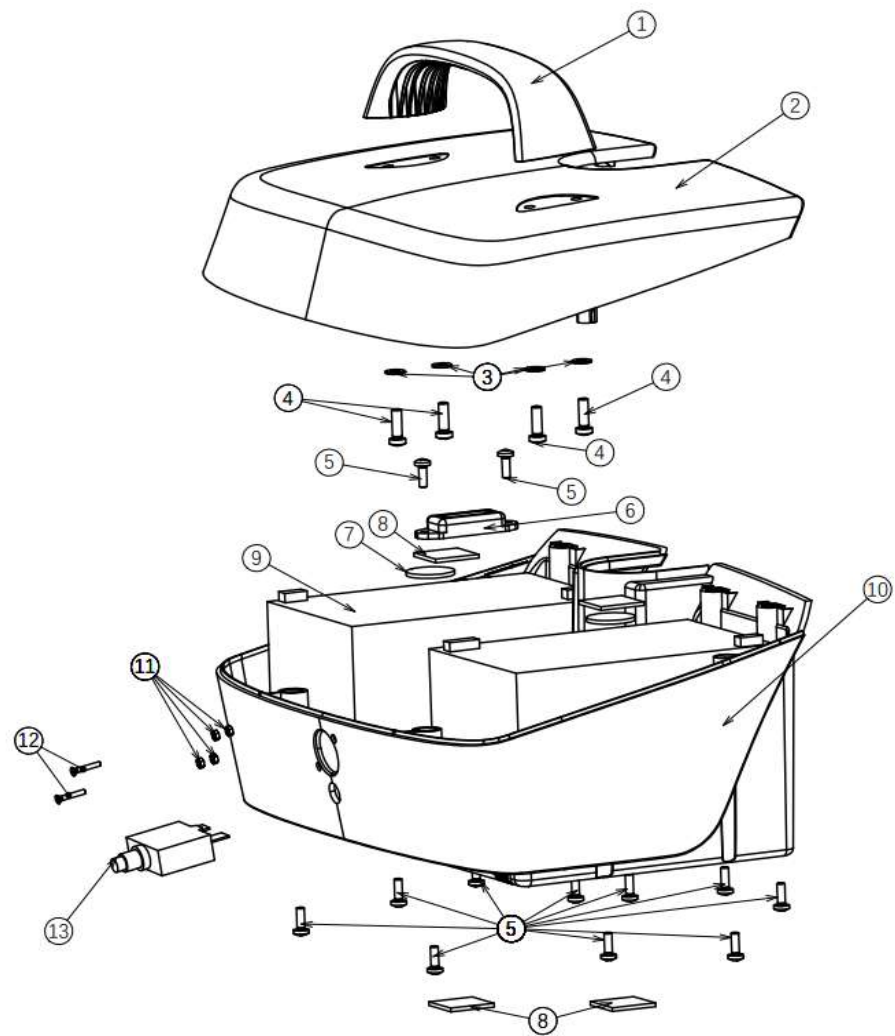

no	part nu	part	QT				
0	1920296	BATTERYBOX COMPLETE	1				
0	1918891	BATTERY BOX WITHOUT BATTERIES	1				
1							
2							
3							
4							
5	1922784	SCREW M4X10	5				
6							
7							
8							
9	1922670	CPX15 -12	2				
10							
11							
12							
13	1920531	MAIN FUSE					
0	1919798	CHARGER 2 A	1	not depicted			
not depicted	1922548	Charger 3A (option)	0				

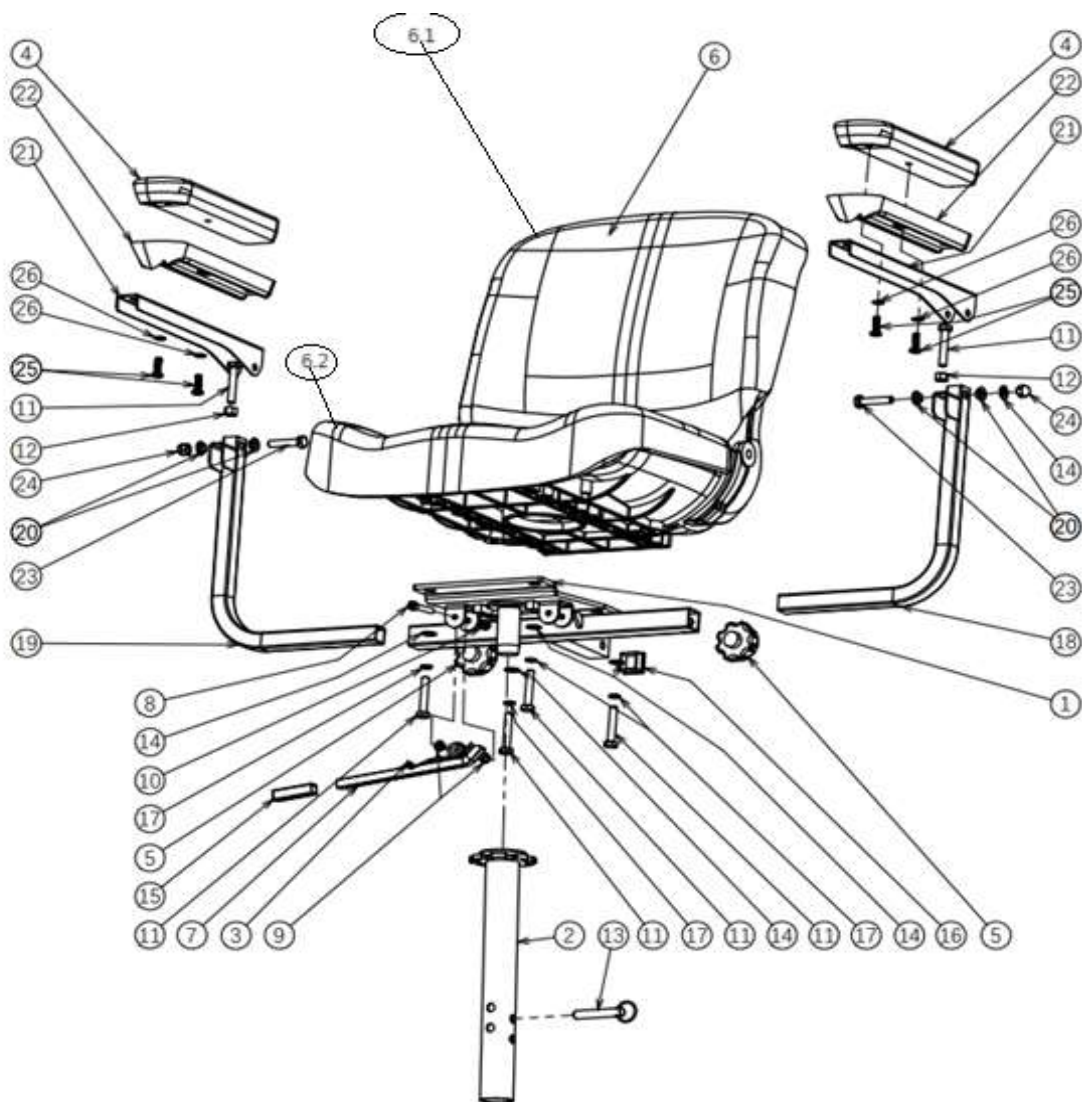


no	part nu	part	QT			
0						
1						
2	1918898	SEAT TUBE	1			
3						
4	1918895	ARM PAD	2			
5	1920266	BOLT ARMREST FIXATION	2			
6	1918899	CHAIR	1			
6.1	1920640	BACK	1			
6.2	1920641	SEAT	1			
7	1920645	SEAT HANDLE	1			
NOT DEPICTED	1923200	SPRING SEAT HANDLE	1			
8						
8						
9						
10						
11	1924043	BOUT M8 X 45 DIN931	4			
12						
13	1922726		1			
14	1924044	RONDSEL M8 DIN126	4			
15			4			
16						
17	1924045	RONDSEL VEER M8DIN127B	4			
18	1918903	ARMREST LOWER LEFT	1			
19	1918907	ARMREST LOWER RIGHT	1			
20						
21	1918931	ARMRESR UPPER PART	2			
22						
23						
24						
	1920830	ARMREST COMPLETE L	1			
	1920831	ARMREST COMPLETE R	1			
	1921438	SEAT PLATE COMP W/A TUBE	1			

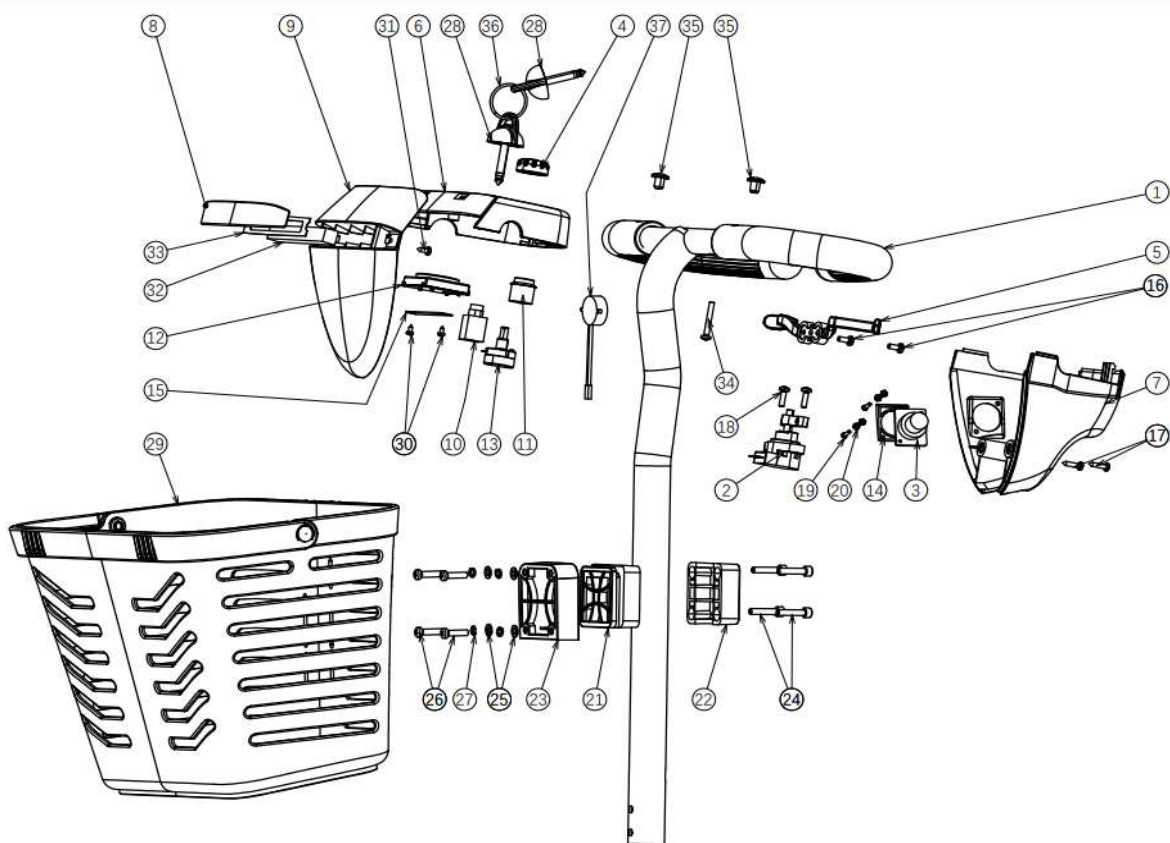

no	part nu	part	QT	
1				
2	1918911	SPEED DIAL ASSEMBLY	1	
3	1918918	CHARGER SOCKET	1	
4	1924415	SPEED KNOB	1	
5	1918910	SPEED LEVER	1	
6	1923868	TOP COVER		
7	1918913	STEERING COVER BACK	1	

8	1921463	FRONT LIGHT COVER	1	
9	1918914	STEERING COVER FRONT	1	
10	1919437	CONTACT IGNITION	1	
11	1920423		1	
12	1918912	BATTERY INDICATOR	1	
13	1918909	POT SPEED	1	
14				
15				
16				
17				
18				
19				
20				
21	1920502	ASSEMBLY PLATE BASKET MIDDLE	1	
22	1920510	ASSEMBLY PLATE BASKET REAR	1	
23	1920501	ASSEMBLY PLATE BASKET FRONT	1	
	1920832	BASKET HOLDER COMPLETE	1	
24	1920511	BOUT BZKT M5X25 DIN912	2	
25	1920504	WASHER	4	
26	1920505	BOLT M5X20	4	
27				
28	1918908	IGNITION KEY	1	
29	1920509	BASKET	1	
30				
31				
32	1920629	FRONT LIGHT	1	
33				
34				
35				
	3990651	MIRROR LEFT	1	
	3990652	MIRROR RICHT	1	
37	1920628	HORN	1	

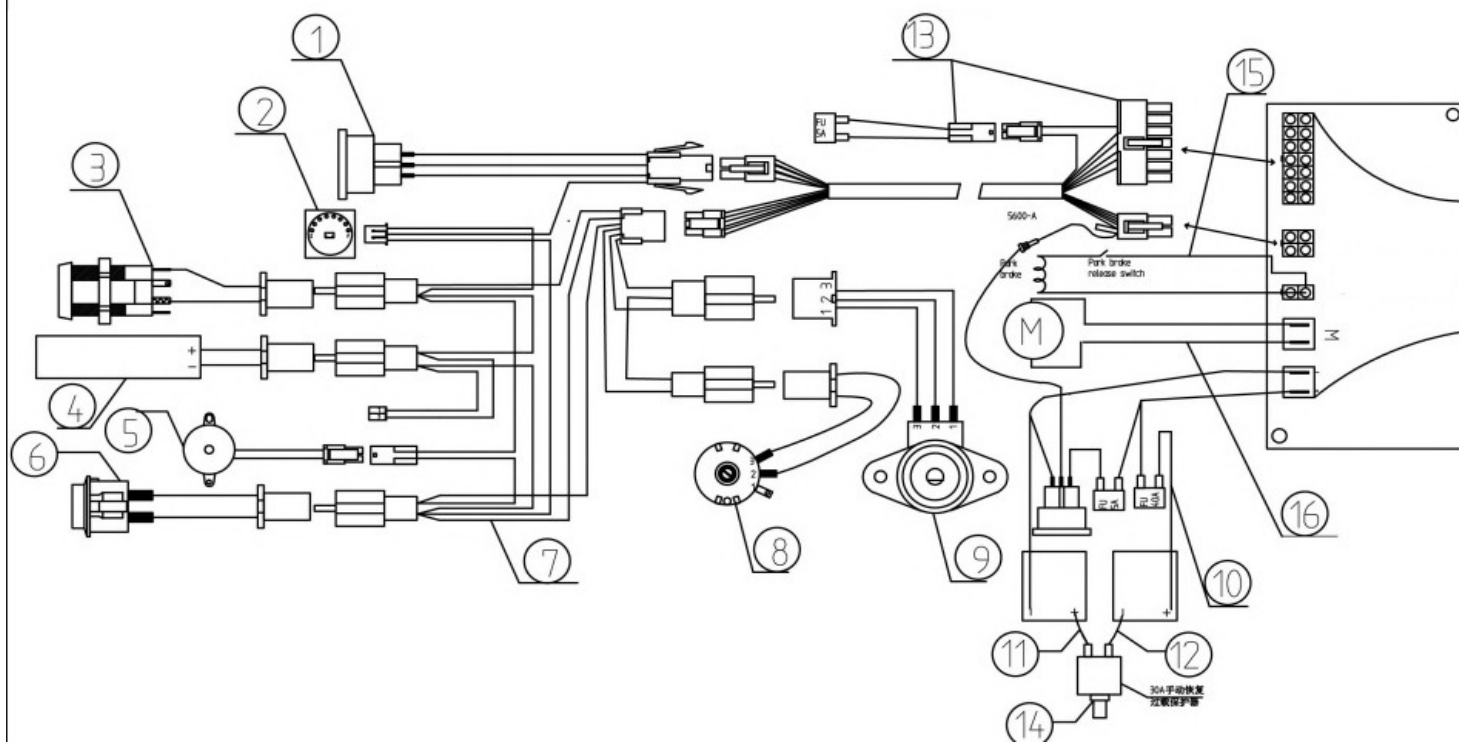

no	part nu	part	QT	
1	1918918	CHARGER SOCKET	1	
2				
3				
4				
5				
6				
7	1918976	CABLE TO PCB	1	
8				
9	1918909	POT. SPEED	1	
10	1921636	CABLE + FUSE	1	
11				
12				
13	1922401	MAIN CABLE	1	
14				
15				
16				