



01.07.2026

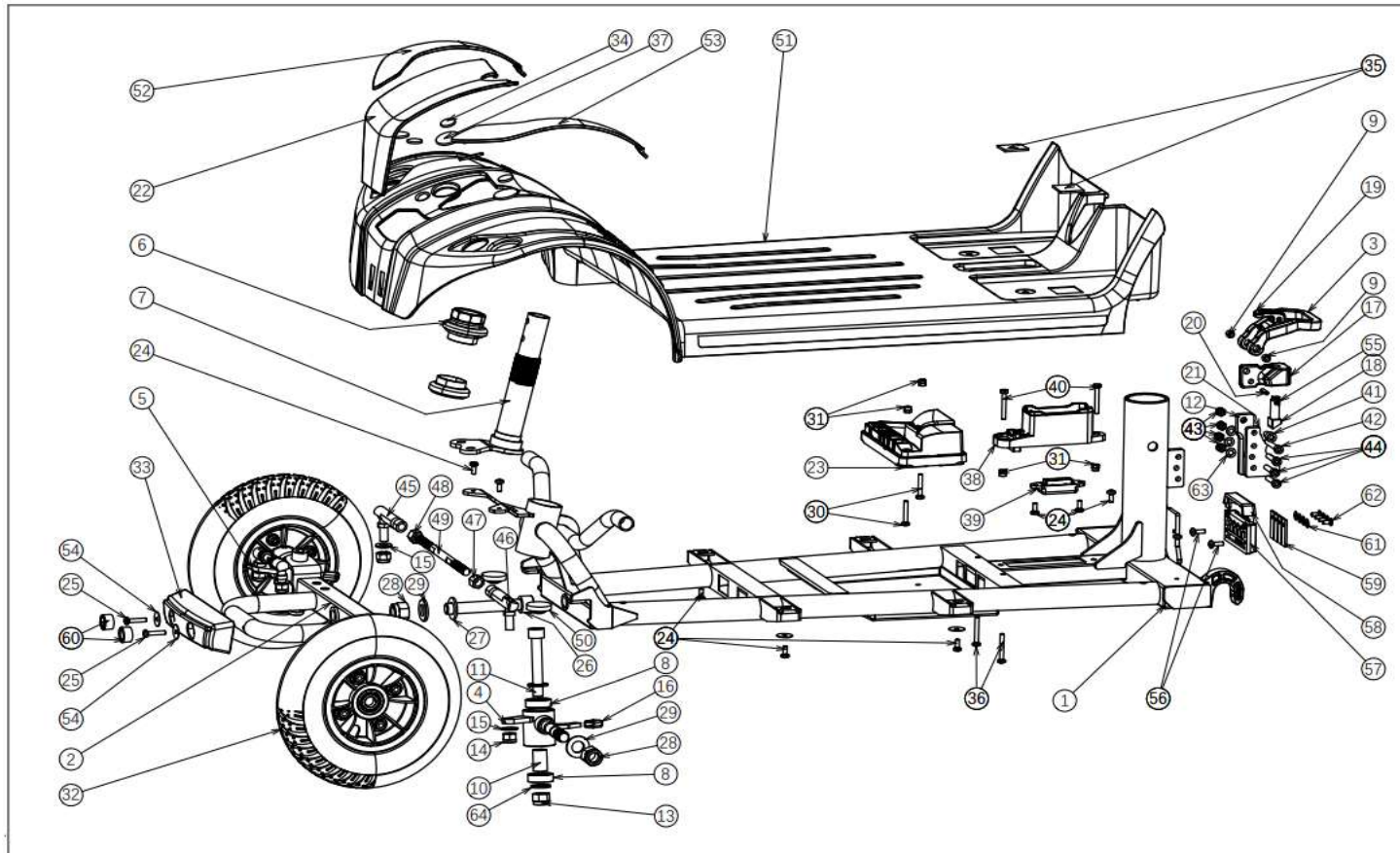


no	part nu	part	QT	no	part nu	part	QT
1	1920656	MAIN FRAME	1	36			
2	1918862	FRONT FORK	1	37			
3				38	1920657	BATTERY CONNECTOR HOLDER 1	1
4	1920340	FRONT AXLE L		39	1920658	BATTERY CONNECTOR HOLDER	1
5	1920341	FRONT AXLE R			1922814	BATTERY CONNECTOR +CABLE	1
6				40			
7	1918856	STEERING AXLE ONE	1	41			
7				42			
7				43			
7				44	1918857	BALL JOINT LE	
8				46	1918858	BALL JOINT RIGHT	
8				45			
8				47			
8				48	1918859	CONNECTION ROD	
9			1	49			
10				50	1918851	FOOT PLATE	1
11				51	1920443	SET COLOR PANEL FRONT green	
				52	1920444	SET COLOR PANEL FRONT orange	
				52	1920445	SET COLOR PANEL FRONT Blue	
				52	1920446	SET COLOR PANEL FRONT pink	
			1	52			
13				53	1920634	WASHER M4 NF E25-513	
14				54			
15			2	55			
16				56			
17				57	1918861	FRONT CONTACT POINT HOLDER	1
18			2	58			
19				59	1918860	CONTACT POINT FRONT	4
20				60			
21				61			
22	1918852	FRONT COVER	1	62			


23	1919090	POWER MODULE	1	59	1920635	BOLT CAP	6
24			1				
25	1920633	BOLT M4X20 DIN 7985	13				
26							
27							
28							
29							
30							
31							
32	1922939	FRONTWHEEL ONE AIR	2				
33	1920632	FRONT BUMPER	1				
34							
35							

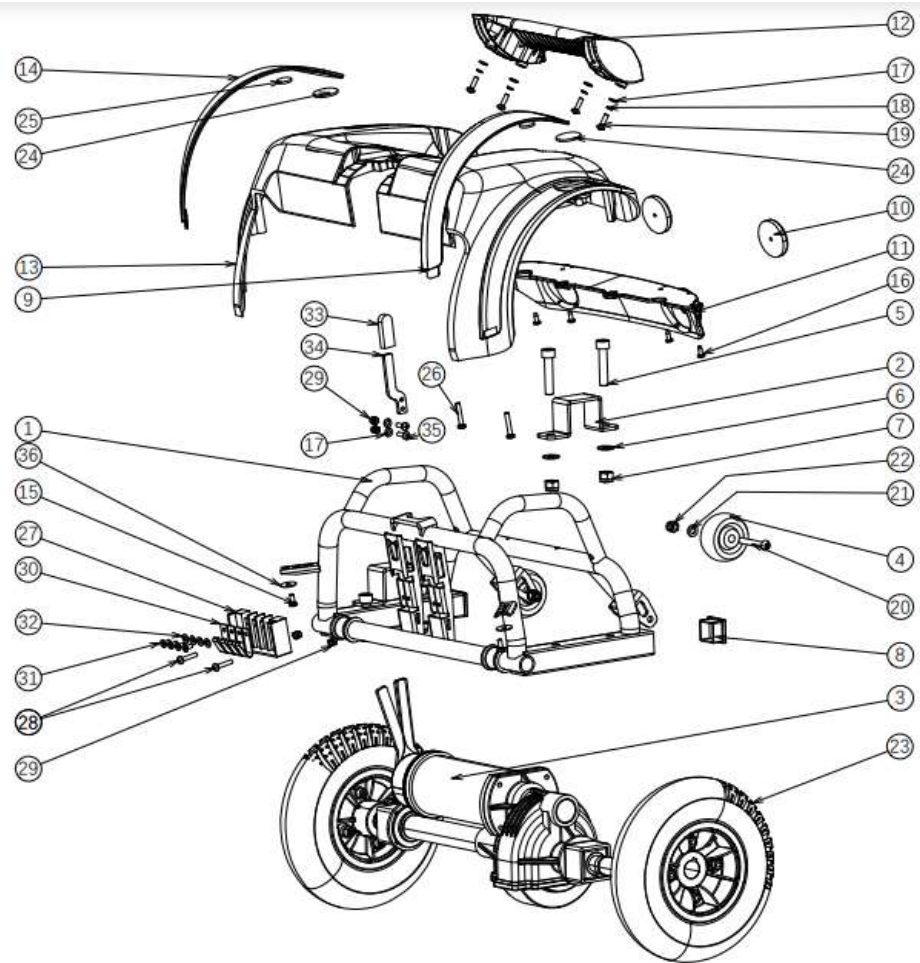


no	part nu	part	QT		part nu	part	QT
0	1920394	TIRE	4				
0	1920393	INNER TIRE	4		1922206	SCREW ST3,5x9,5	4
1				16	1922402	WASHER M4	4
2				17	1922403	SPRING WASHER	4
3				18	1922404	SCREW M4	4
3	1918887	DIFF.+MOTOR+REM	1	19	1922549	BOLT ANTI TIP	2
3	1920395	DIFF	1	20			
not depicted	1922014	BUSH TRANSMISION .	1				
3	1920396	MOTOR	1				
3	1920397	MOTORBRAKE	1				
4	1922523	WHEEL ANTI TIPPING	2		1922550	SPRING WASHER	2
4				21	1922551	LOCK NUT	2
4				22			
4				23	1922938	REAR WHEEL LU	2
5				24			
6				25			
7				26			1
8				27	1918889	REAR CONTACT POINT HOLDER	1
9				28			
10	1921454	REAR RED REFLECTOR	2	30	1918890	CONTACT POINT REAR	4
11	1918882	REAR COVER	1				
12	1918886	HANDLE REAR	1	31			
13	1918853	COVER MIDDLE	1	32			
14	1920439	COLOR PANEL SET L/R green		33			
14	1920440	COLOR PANEL SET L/R orange		34			
14	1920441	COLOR PANEL SET L/R Pink					
14	1920442	COLOR PANEL SET L/R blue					
15							
16				35			
17	1922190	RED MOTORBRAKE LEVER COVER	1	36			
not depicted	1923143	AXLE KEY	1				

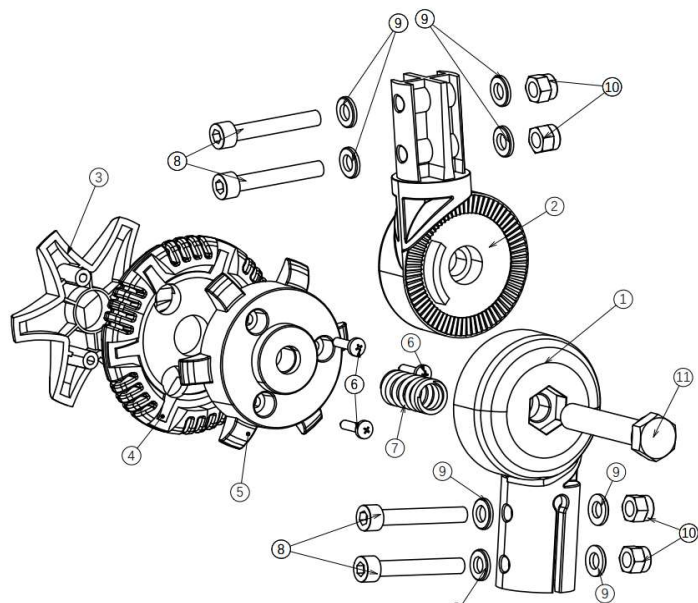


no	part nu	part	QT				
1	1921317	lower parrrt angle adujstment					
2							
3	1921320	outer knob angle adjust	1				
4	1921319	middle knob angle adjust	1				
5	1921318	inner knob angle adjust	1				
6	1921322	bolt M5 X 16					
7	1921321	spring knob angle adjust	1				
8							
9							
10							
11	1922522	bolt angle adjust M10 X55	1				
not depicted	1922514	Set 1/12	1				

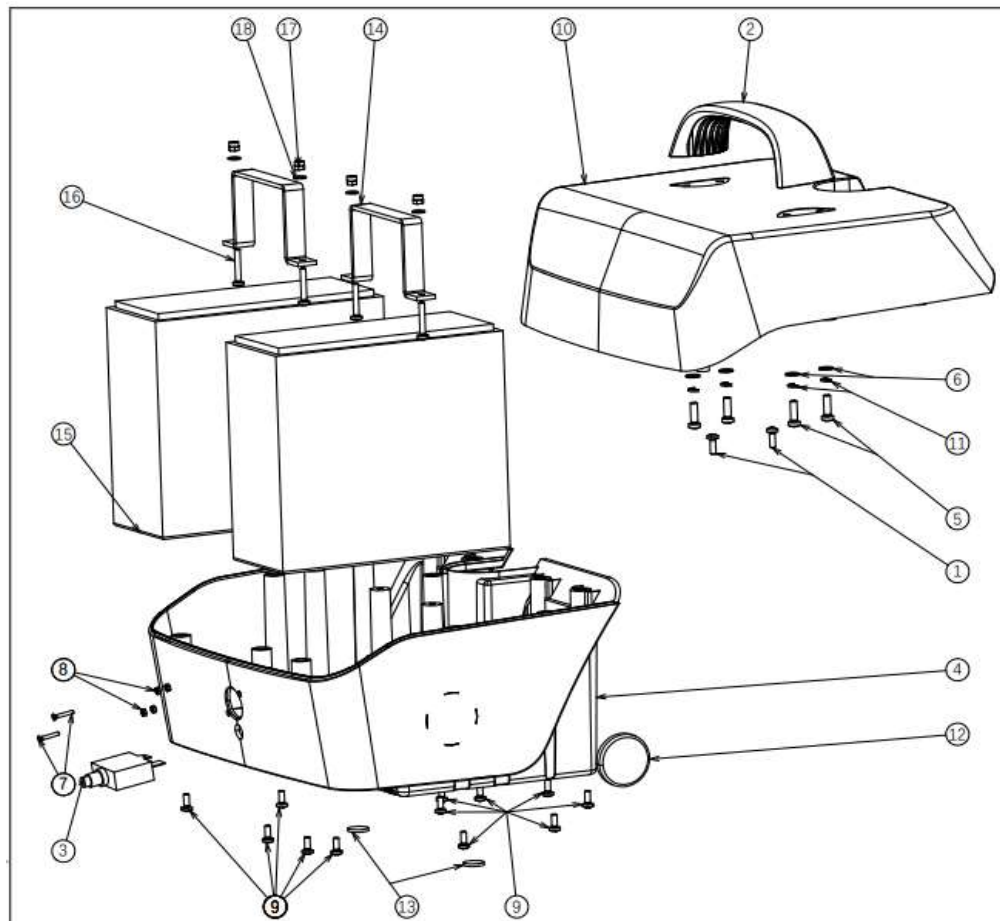

no	part nu	part	QT				
0	1920391	BATTERYBOX 25 AHCOMPLETE	1				
0	1920346	BATTERY BOX 25AH WITHOUT BATTERIES	1				
1							
2							
3	1920531	MAIN FUSE					
4							
5							
6							
7							
8							
9	1922784	Screw M4x10	5				
10							
11							
12							
13							
14							
15	1920301	BATTERY 25 A/H	2				
16							
17							
18							
	1922548	CHARGER 3 amp	optie				
0	1919798	CHARGER 2A	1				
not depicted	1922548	CHARGER 3 A (option)	0				

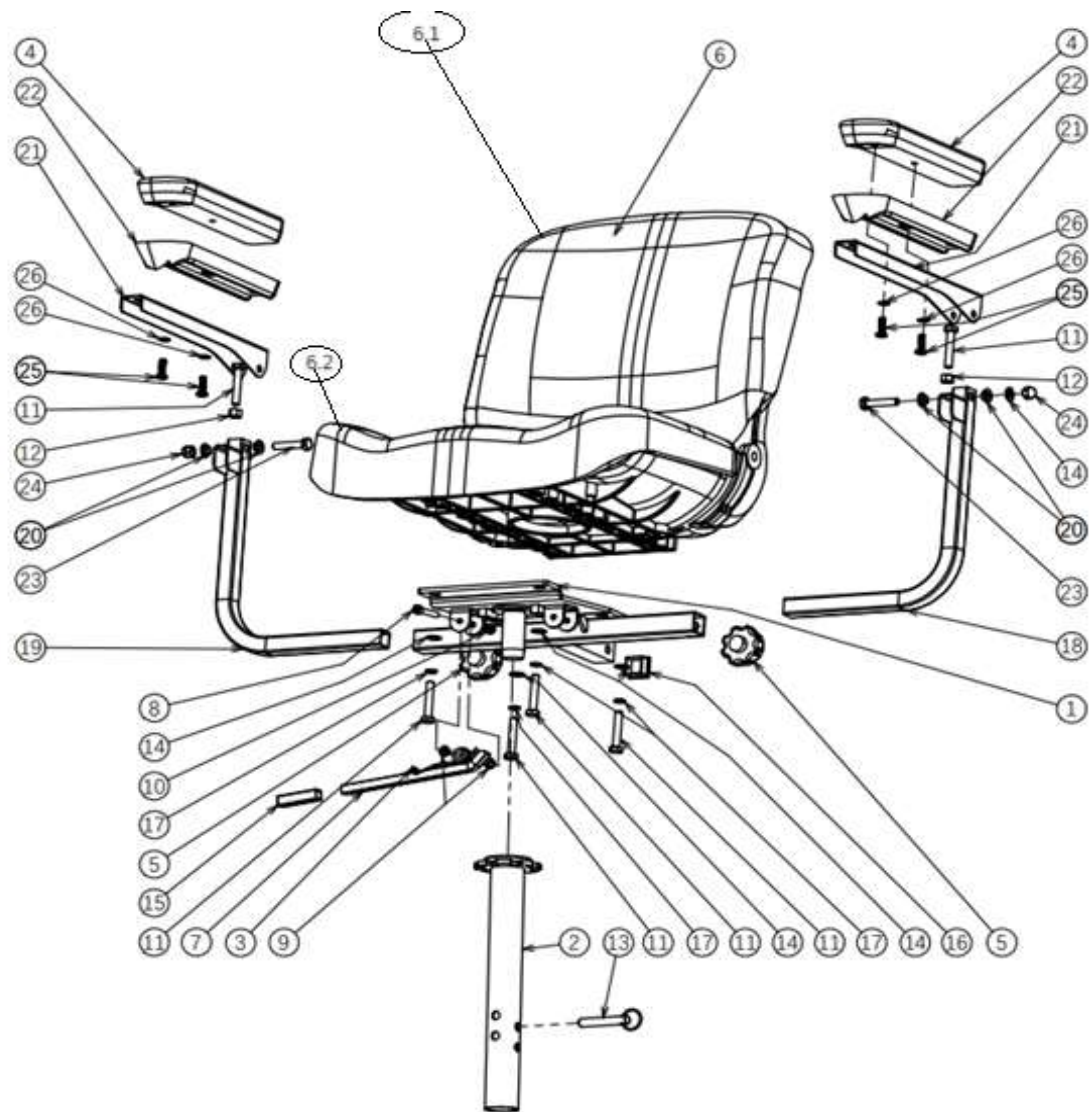


no	part nu	part	QT			
0						
1						
2	1918898	SEAT TUBE	1			
3						
4	1918895	ARM PAD	2			
5	1920266	BOLT ARMREST FIXATION	2			
6	1918899	CHAIR	1			
6.1	1920640	BACK	1			
6.2	1920641	SEAT				
7	1920645	SEAT HANDLE	1			
not depicted	1923200	SPRING SEAT HANDLE	1			
8						
8						
9						
10						
11	1924043	BOUT M8 X 45 DIN931	4			
12						
13						
14	1924044	RONDSEL M8 DIN126	4			
15						
16						
17	1924045	RONDSEL VEER M8DIN127B	4			
18	1918903	ARMREST LOWER LEFT	1			
19	1918907	ARMREST LOWER RIGHT	1			
20						
21	1918931	ARMRESR UPPER PART	2			
22						
23						
24						
	1920830	ARM REST COMPLETE L	1			
	1920831	ARM REST COMPLETE R	1			
1921438 SEAT PLATE COMPL W/A tube						

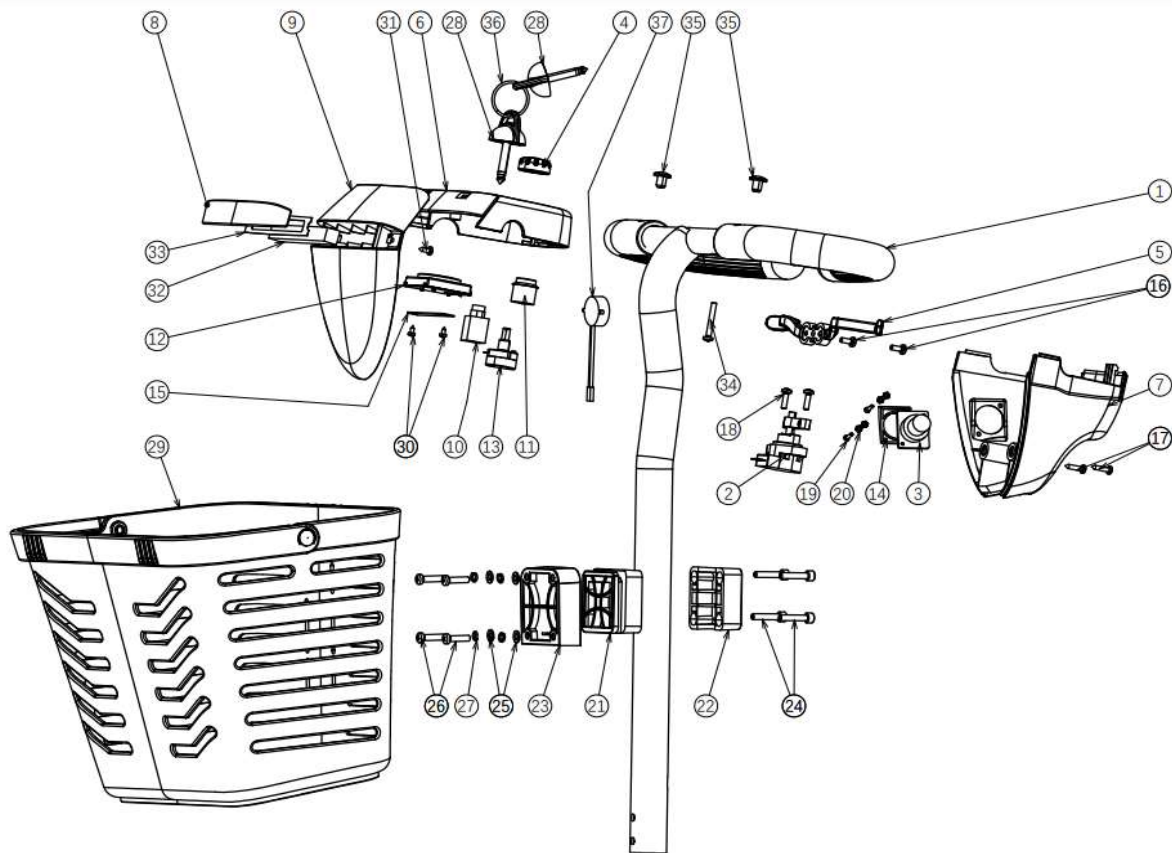

no	part nu	part	QT
1			
2	1918911	SPEED DIAL ASSEMBLY	1
3	1918918	CHARGER SOCKET	1
4	1924415	SPEED KNOB	1
5	1918910	SPEED LEVER	1
6	1923868	TOP COVER	1

7	1918913	STEERING COVER BACK	1	
8	1921463	FRONT LIGHT COVER	1	
9	1918914	STEERING COVER FRONT	1	
10	1919437	CONTACT IGNITION	1	
11	1920423	HORN SWITCH	1	
12	1918912	BATTERY INDICATOR	1	
13	1918909	POT SPEED	1	
14				
15				
16				
17				
18				
19				
20				
21	1920502	ASSEMBLY PLATE BASKET MIDDLE	1	
22	1920510	ASSEMBLY PLATE BASKET REAR	1	
23	1920501	ASSEMBLY PLATE BASKET FRONT	1	
	1920832	BASKET HOLDER COMPLETE		
24	1920511	BOLT M5X25	2	
25	1920503	WASHER	4	
26	1920505	BOLT M5X20	4	
27				
28	1918908	IGNITION KEY	1	
29	1920509	BASKET	1	
30				
31				
32	1920629	FRONT LIGHT	1	
33				
34				
35				
	3990651	MIRROR LEFT	1	
	3990652	MIRROR RICHT	1	
37	1920628	HORN	1	

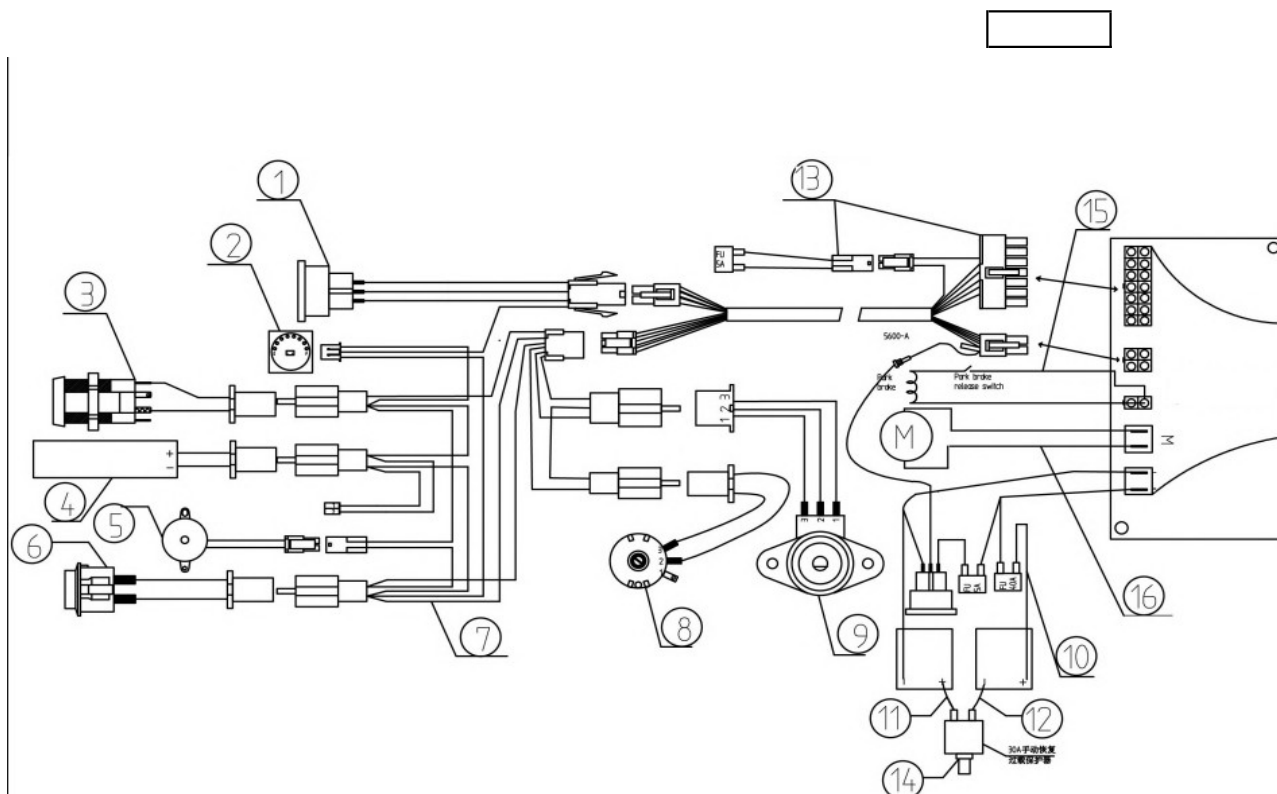

no	part nu	part	QT
1	1918918	CHARGER SOCKET	1
2			
3			
4			
5			
6			
7	1918976	SHORT CABLE TILLER	1
8			
9	1918909	POT. SPEED	1

10	1921636	CABLE+ FUSE	1	
11				
12				
13	1922401	MAIN CABLE	1	
14				
15				
16				