


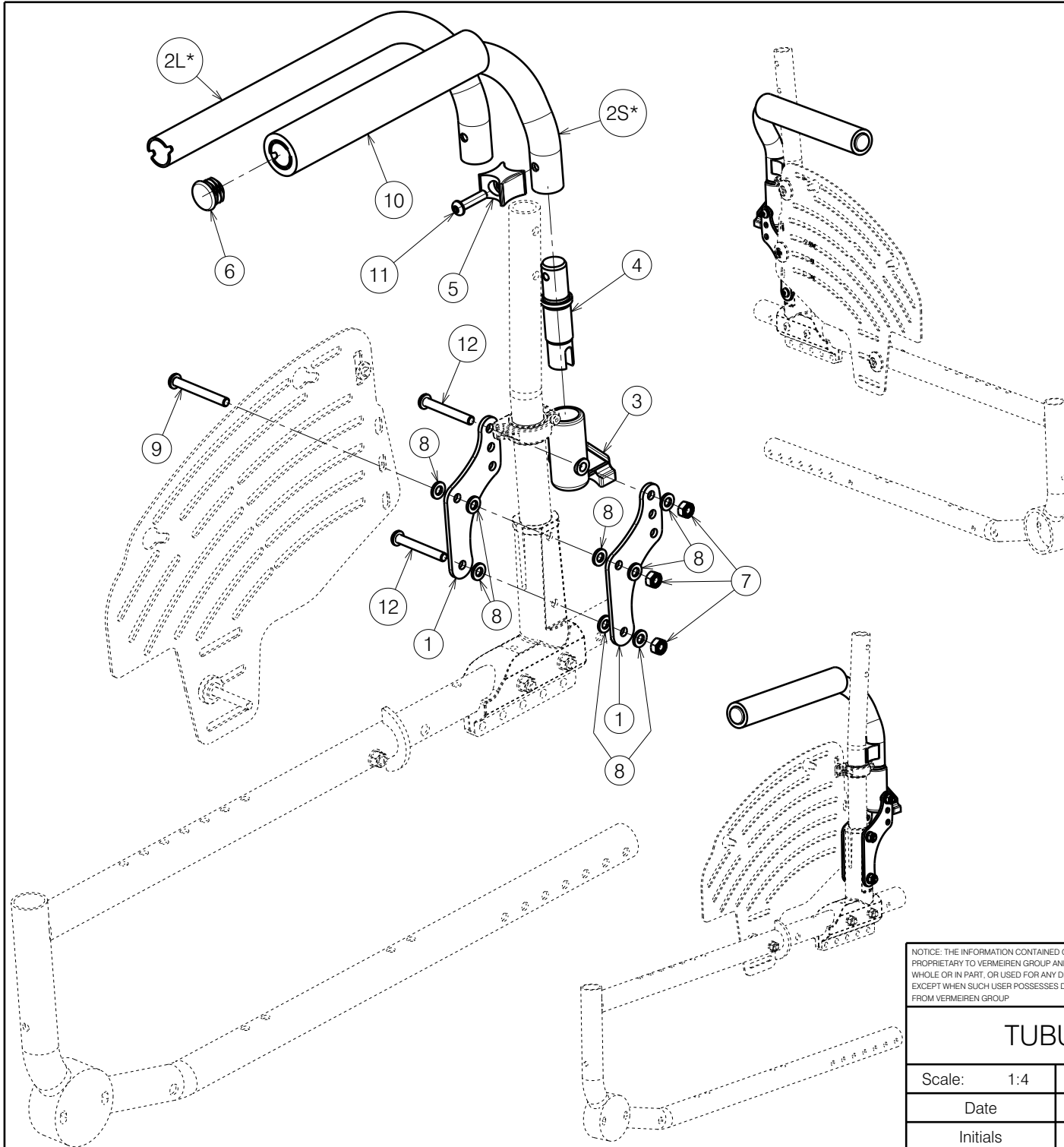

No.	Plan	Article	Qty
1	P00118357	2920865	4
2S*	YP02372(R)	1899865	1
	YP02373(L)	1899765	1
2L*	YP07780(R)	2781965	1
	YP07781(L)	2782065	1
3	XP00335	2990264	2
4	XP00615 (S)	2990920	2
	XP03059 (L)	2996852	2
5	XP01843 (Ø22)	2993563	2
6	CAP Ø22	1990648	2
7	M6 DIN985 (Black)	2994063	6
8	M6 DIN125 (Black)	2994068	16
9	M6x50 ISO7380 (Black)	2994965	2
10	X003135	1991107	0,38m
11	M6x25 ISO7380 (Black)	2994062	2
12	M6x45 ISO7380 (Black)	2994086	4

* - L - Long
S - Short

- Only right side depicted,
- Quantities for left and right sides

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

TUBULAR ARMREST / V500 ACT				---
				Plan No. E00121056/A1.1
Scale: 1:4	Drafted	Verified	Confirmed	
Date	21.01.2026	21/01/2026	21/01/2026	
Initials	AZ	AS	AS	